

## Overview

### Useful For

A marker of clonality and B-cell lineage

### Reflex Tests

Test ID	Reporting Name	Available Separately	Always Performed
IHTOI	IHC Initial, Tech Only	No	No
IHTOA	IHC Additional, Tech Only	No	No

## Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

### Method Name

Immunohistochemistry (IHC)

### NY State Available

Yes

## Specimen

### Specimen Type

TECHONLY

### Advisory Information

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

### Shipping Instructions

Attach the green pathology address label and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

### Specimen Required

**Supplies:** Immunostain Technical Only Envelope (T693)

**Specimen Type:** Tissue

**Container/Tube:** [Immunostain Technical Only Envelope](#)

**Preferred:** 2 Unstained positively charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick.

**Acceptable:** Formalin-fixed, paraffin-embedded (FFPE) tissue block

### Digital Image Access

- Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: [www.mayocliniclabs.com/test-info/ihc/index.html](http://www.mayocliniclabs.com/test-info/ihc/index.html)
- Clients ordering stains using a manual requisition form will not have access to digital images.
- Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: [www.mayocliniclabs.com/test-info/ihc/faq.html](http://www.mayocliniclabs.com/test-info/ihc/faq.html)

### Forms

If not ordering electronically, complete, print, and send a [Immunohistochemical \(IHC\)/In Situ Hybridization \(ISH\) Stains Request](#) (T763) with the specimen.

### Reject Due To

Wet/frozen tissue Cytology smears Nonformalin fixed tissue Nonparaffin embedded tissue Noncharged slides ProbeOn slides	Reject
--	--------

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

## Clinical and Interpretive

### Clinical Information

Kappa or lambda immunoglobulin light chains pair with immunoglobulin heavy chains to form complete immunoglobulin molecules. These proteins serve as receptors for antigens in B lymphocytes and are the secretory products of plasma cells, forming the humoral arm of the immune system. Because individual B cells or plasma cells synthesize immunoglobulin containing either kappa or lambda light chains, but not both, immunoperoxidase stains for light chains can be applied to lymphocyte and plasma cell populations as a marker of clonality and B-cell lineage.

### Interpretation

This test does not include pathologist interpretation, only technical performance of the stain. If an interpretation is required, order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity and documentation is retained at Mayo Clinic Rochester. If a control tissue is not included on the slide, a scanned image of the relevant quality control tissue is available upon request, call 855-516-8404.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

## Cautions

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen dependent. Best practice is for paraffin sections to be cut within 6 weeks.

## Clinical Reference

1. Peterson LC, Brown BA, Crosson JT, Mladenovic J: Application of the immunoperoxidase technique to bone marrow trephine biopsies in the classification of patients with monoclonal gammopathies. Am J Clin Pathol. 1986;85:688-693

2. Taylor CR, Burns J: The demonstration of plasma cells and other immunoglobulin-containing cells in formalin-fixed, paraffin-embedded tissues using peroxidase labeled antibody. J Clin Pathol. 1974;27:14-20

## Performance

### Method Description

Immunohistochemistry on sections of paraffin-embedded tissue.(Cartun RW, Taylor CR, Dabbs DJ: Techniques of immunohistochemistry: Principles, pitfalls, and standardization. In: Dabbs DJ, ed. Diagnostic Immunohistochemistry. 5th ed. Elsevier; 2019:1-46)

### PDF Report

No

### Day(s) and Time(s) Test Performed

Monday through Friday

### Analytic Time

1 day

### Maximum Laboratory Time

3 days

### Specimen Retention Time

Until staining is complete

### Performing Laboratory Location

Rochester

## Fees and Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

### Test Classification

This test has been cleared, approved or is exempt by the U.S. Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

---

**CPT Code Information**

88342-TC, primary

88341-TC, if additional IHC

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
KAIHC	Kappa IHC, Tech Only	Order only;no result

Result ID	Test Result Name	Result LOINC Value
70794	Kappa IHC, Tech Only	Bill only; no result