

**Overview**
**Reflex Tests**

Test ID	Reporting Name	Available Separately	Always Performed
FAMCS	Amphetamines Confirmation, S	No	No
FBBCS	Barbiturates Confirmation, S	No	No
FBCS	Benzodiazepines Confirmation, S	No	No
FCANS	Cannabinoids Confirmation, S	No	No
FCMC	Cocaine and Metabolites Confirm, S	No	No
FMMCS	Methadone and Metabolite Confirm, S	No	No
FOFU	Opiates-Free Unconjugated Conf, S	No	No
FPCCS	Phencyclidine Confirmation, S	No	No

**Testing Algorithm**

If appropriate, confirmation testing will be performed at an additional charge.

**Method Name**

Enzyme-Linked Immunosorbent Assay (ELISA)

**NY State Available**

Yes

**Specimen**
**Specimen Type**

Serum Red

**Specimen Required**

**Specimen Type:** Serum

**Container/Tube:** Red top

**Specimen Volume:** 4 mL

**Collection Instructions:** Â Draw blood in a plain, red-top tube(s), serum gel tube is not acceptable. Spin down and

send 4 mL of serum refrigerated in plastic, preservative-free vial.

### Specimen Minimum Volume

2 mL

### Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	Polymer gel separation tube (SST or PST).

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	14 days	

### Clinical and Interpretive

### Reference Values

Reporting limit determined each analysis.

Test	Result	Units
Opiates	None Detected	ng/mL
Cocaine/Metabolites	None Detected	ng/mL
Benzodiazepines	None Detected	ng/mL
Cannabinoids	None Detected	ng/mL
Amphetamines	None Detected	ng/mL
Barbiturates	None Detected	ng/mL
Methadone/Metabolite	None Detected	ng/mL
Phencyclidine	None Detected	ng/mL
Methamphetamine/MDMA	None Detected	ng/mL
Oxycodone/Oxymorphone	None Detected	ng/mL

### Performance

### PDF Report

No

**Day(s) and Time(s) Test Performed**

Monday through Sunday

**Analytic Time**

4 - 8 days

**Maximum Laboratory Time**

6 - 12 days

**Performing Laboratory Location**

NMS Labs

**Fees and Codes**
**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

80307

**LOINC® Information**

Test ID	Test Order Name	Order LOINC Value
FD10S	Drugs of Abuse Screen (10 Panel), S	Not Provided

Result ID	Test Result Name	Result LOINC Value
Z5245	Opiates	8219-8
Z5246	Cocaine/Metabolites	73823-7
Z5247	Benzodiazepines	46976-7
Z5248	Cannabinoids	8172-9
Z5249	Amphetamines	8149-7
Z5250	Barbiturates	20421-4
Z5251	Methadone/Metabolite	59705-4
Z5252	Phencyclidine	8236-2
Z5253	Methamphetamine/MDMA	91034-9
Z5254	Oxycodone/Oxymorphone	77732-6