

Overview**Method Name**

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

NY State Available

Yes

Specimen**Specimen Type**

Varies

Specimen Required**Submit only 1 of the following specimens:****Plasma**

Specimen Type: Plasma

Container/Tube: Green Top

Specimen Volume: 2 mL

Collection Instructions: Draw blood in a green-top (sodium heparin) tube, **plasma gel tube is not acceptable**. Spin down and send 2 mL of sodium heparin plasma refrigerated in a plastic vial.**Serum**

Specimen Type: Serum

Container/Tube: Red

Specimen Volume: 2 mL

Collection Instructions: Draw blood in a plain, red-top tube, **serum gel tube is not acceptable**. Spin down and send 2 mL of serum refrigerated in a plastic vial.**Specimen Minimum Volume**

0.4 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icteric	NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	7 days	
	Frozen	30 days	
	Ambient	72 hours	

Clinical and Interpretive**Reference Values**

Reference Range: 10 - 30 ng/mL

Performance**PDF Report**

No

Day(s) and Time(s) Test Performed

Monday through Sunday

Analytic Time

3 days

Maximum Laboratory Time

5 - 7 days

Specimen Retention Time

2 weeks

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees and Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

CPT Code Information

80369

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FFLEX	Cyclobenzaprine (Flexeril)	6797-5

Result ID	Test Result Name	Result LOINC Value
Z1130	Cyclobenzaprine (Flexeril)	6797-5