

Overview**Method Name**

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

NY State Available

Yes

Specimen**Specimen Type**

Varies

Specimen Required

Submit only 1 of the following specimens:

Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 2 mL sodium heparin plasma frozen in plastic vial.

Serum

Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 2 mL of serum frozen in plastic vial.

Specimen Minimum Volume

0.3 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Frozen	180 days	

Clinical and Interpretive**Reference Values**

Flurazepam:

Reference Range: 0 - 30 ng/mL

Desalkylflurazepam:

Reference Range: 30 - 150 ng/mL

Flurazepam + Desalkylflurazepam:

Reference Range: 30 - 180 ng/mL

Performance

PDF Report

No

Day(s) and Time(s) Test Performed

Monday through Sunday

Analytic Time

5 days

Maximum Laboratory Time

7 - 9 days

Specimen Retention Time

2 weeks

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees and Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

CPT Code Information

80346

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FFLUR	Flurazepam (Dalmane)	73826-0



Result ID	Test Result Name	Result LOINC Value
Z1133	Flurazepam	46979-1
Z1160	Desalkylflurazepam	46980-9
Z1161	Flurazepam + Desalkylflurazepam	73826-0