

Overview**Method Name**

High-pressure liquid chromatography/tandem mass spectrometry (HPLC/MS-MS)

NY State Available

Yes

Specimen**Specimen Type**

Serum

Specimen Required

Draw blood in a plain, red-top tube(s). Separate serum within an hour. Spin down and send 3 mL of serum frozen in a plastic vial.

Note: Serum gel tube is acceptable, but must pour off into a plastic vial.

Specimen Minimum Volume

1 mL Note: Does not allow for repeat analysis.

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	365 days	
	Refrigerated	14 days	
	Ambient	6 days	

Clinical and Interpretive**Reference Values**

Units = ng/dL

Performance

PDF Report

No

Day(s) Performed

Monday

Report Available

3 to 20 days

Performing Laboratory Location

Esoterix Endocrinology

Fees and Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their Regional Manager. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

CPT Code Information

80299

LOINC® Information

Test ID	Test Order Name	Order LOINC Value
FDXM	Dexamethasone	14062-4

Result ID	Test Result Name	Result LOINC Value
Z1745	Dexamethasone	14062-4