



# Test Definition: FANSE

Annatto Seed (Bixa orellana) IgE

## Overview

### Method Name

Enzyme Immunoassay (EIA)

### NY State Available

Yes

## Specimen

### Specimen Type

Serum

### Specimen Required

#### Collection Container/Tube:

**Preferred:** Red top

**Acceptable:** Serum gel

**Submission Container/tube:** Plastic vial

**Specimen Volume:** 0.5 mL Serum

#### Collection Instructions:

1. Centrifuge and aliquot serum into a plastic vial.
2. Send refrigerated.

### Specimen Minimum Volume

See Specimen Required

### Reject Due To

|               |        |
|---------------|--------|
| Gross lipemia | Reject |
|---------------|--------|

### Specimen Stability Information

| Specimen Type | Temperature              | Time    | Special Container |
|---------------|--------------------------|---------|-------------------|
| Serum         | Refrigerated (preferred) | 28 days |                   |
|               | Ambient                  | 28 days |                   |
|               | Frozen                   | 28 days |                   |

## Clinical & Interpretive

### Clinical Information

Refer to [www.eurofins-viracor.com/test-menu/](http://www.eurofins-viracor.com/test-menu/)

**Reference Values**

<0.35 kU/L

**Interpretation**

| Class | IgE (kU/L)  | Comment              |
|-------|-------------|----------------------|
| 0     | <0.35       | Below Detection      |
| 1     | 0.35-0.69   | Low Positive         |
| 2     | 0.70-3.49   | Moderate Positive    |
| 3     | 3.50-17.49  | Positive             |
| 4     | 17.50-49.99 | Strong Positive      |
| 5     | >49.99      | Very Strong Positive |

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

4 to 7 days

**Performing Laboratory Location**

Eurofins Viracor

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

---

86003

**LOINC® Information**

| Test ID | Test Order Name  | Order LOINC® Value |
|---------|------------------|--------------------|
| FANSE   | Annatto Seed IgE | 50638-6            |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z3074     | Annatto Seed IgE | 50638-6             |
| Z3075     | Class            | 103035-2            |