

# **Test Definition: FHSSE**

Helminthosporium sativum/Drechslera IgE

# **Overview**

#### **Method Name**

Enzyme Immunoassay (EIA)

#### **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Serum

# **Specimen Required**

Specimen Type: Serum
Container/Tube: Red or SST
Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5

mL of serum refrigerated in a plastic vial.

#### **Specimen Minimum Volume**

0.5 mL

# **Reject Due To**

Gross Lipemia	Mild OK; Gross reject

# **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	365 days	

# **Clinical & Interpretive**

#### **Clinical Information**

This assay is used to detect allergen specific-IgE using an enzyme immunoassay (EIA). In vitro allergy testing is the primary testing mode for allergy diagnosis.



# **Test Definition: FHSSE**

Helminthosporium sativum/Drechslera IgE

#### **Reference Values**

<0.35 kU/L

# Interpretation

<u>Class</u>	IgE (kU/L)	<u>Comment</u>
0	<0.35	<b>Below Detection</b>
1	0.35-0.69	Low Positive
2	0.70-3.49	Moderate Positive
3	3.50-17.49	Positive
4	17.50-49.99	Strong Positive
5	>=50	Very Strong Positive

#### **Performance**

#### **Method Description**

Allergens are covalently coupled to the cellulose paper discs via the APT method. Alkaline phosphatase (AP) labelled anti-IgE is used to quantify the patient's specific IgE.

#### **PDF Report**

No

#### Day(s) Performed

Monday through Friday

#### Report Available

4 to 7 days

# **Performing Laboratory Location**

**Eurofins Viracor** 

# **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.



# **Test Definition: FHSSE**

Helminthosporium sativum/Drechslera IgE

# **CPT Code Information**

86003

# **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FHSSE	Helminthosporium sativum IgE	11175-7

Result ID	Test Result Name	Result LOINC® Value
Z3092	Helminthosporium sativum IgE	11175-7
Z3093	Class	102753-1