

Test Definition: FPHAS

Phadiatop (Allergy Screen)

Overview

Method Name

Enzyme Immunoassay

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Specimen Type: Serum
Container/Tube: Red or SST
Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5

mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

Lipemia	Reject

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	365 days	

Clinical & Interpretive

Reference Values

Negative



Test Definition: FPHAS

Phadiatop (Allergy Screen)

Interpretation

The Phadia Phadiatop test is an allergy screening test with excellent sensitivity and specificity for inhalant allergy. It uses an ImmunoCAP with a balanced mixture of representative allergens, including grasses, trees, weeds, cat, dog, mites and molds. A positive result indicates that the patient is allergic to one or more of these allergens; a negative indicates the patient is not allergic to inhalant allergens. Note that the test does not assess a patient's sensitivity to food, drug, chemical or certain unusual or rare allergens.

Performance

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

3 to 6 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes

Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

CPT Code Information

86005

LOINC® Information

Test ID Te	est Order Name	Order LOINC® Value
FPHAS Ph	Phadiatop (Allergy Screen)	57036-6

Result ID	Test Result Name	Result LOINC® Value
FPHAS	Phadiatop (Allergy Screen)	57036-6