

# **Test Definition: FIFNY**

Interferon-gamma (IFN-y) Serum

## **Overview**

### **Method Name**

Multiplex array electrochemiluminescence

#### **NY State Available**

Yes

## **Specimen**

## **Specimen Type**

Serum

## **Specimen Required**

Specimen Type: Serum

**Collection Container/Tube:** Red or SST **Submission Container/Tube:** Plastic vial

Specimen Volume: 1 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1 mL

of serum frozen in a plastic vial.

### **Specimen Minimum Volume**

1 mL

## **Reject Due To**

Thawing:	Warm reject; Cold OK
----------	----------------------

## **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Frozen	365 days	

## **Clinical & Interpretive**

### **Reference Values**

<8.6 pg/mL



# **Test Definition: FIFNY**

Interferon-gamma (IFN-y) Serum

### **Performance**

### **PDF Report**

No

### Day(s) Performed

Monday, Wednesday, Friday

### **Report Available**

5 to 11 days

## **Performing Laboratory Location**

**Eurofins Viracor** 

### **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

## **Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

### **CPT Code Information**

83520

### **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FIFNY	IFN-y, Serum	27415-9

Result ID	Test Result Name	Result LOINC® Value
FIFNY	IFN-y, Serum	27415-9