



# Test Definition: FABP2

Allergic Bronchopulmonary Aspergillosis Panel

II

## Overview

### Method Name

Enzyme Immunoassay (FEIA)  
Ouchterlony Gel Immunodiffusion

### NY State Available

No

## Specimen

### Specimen Type

Serum

### Specimen Required

#### Collection Container/Tube:

**Preferred:** Red top

**Acceptable:** Serum gel

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 1.5 mL Serum

#### Collection Instructions:

1. Centrifuge and aliquot 1.5 mL of serum into a plastic vial.
2. Send refrigerate.

### Specimen Minimum Volume

Serum: 1 mL

### Reject Due To

Lipemia	Reject
---------	--------

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	7 days	
	Frozen	365 days	

## Clinical & Interpretive

---

**Clinical Information**

<https://www.eurofins-viracor.com/test-menu/>

**Reference Values**

Total IgE (IU/mL):

1-11 months: 0-12

1 year: 0-15

2 years: 1-29

3 years: 4-35

4 years: 2-33

5 years: 8-56

6 years: 3-95

7 years: 2-88

8 years: 5-71

9 years: 3-88

10 years: 7-110

11-14 years: 7-111

15-19 years: 6-96

20-30 years: 4-59

31-50 years: 5-79

51-80 years: 3-48

Aspergillus fumigatus IgE

<0.35 kU/L

Aspergillus fumigatus IgG

<46 mcg/mL

Aspergillus fumigatus Mix Gel Diffusion

Negative

**Interpretation**

Aspergillus fumigatus IgE

Class	IgE (kU/L)	Comment
0	<0.10	Negative
0/1	0.10-0.34	Equivocal/Borderline
1	0.35-0.69	Low Positive
2	0.70-3.49	Moderate Positive
3	3.50-17.49	High Positive
4	17.50-49.99	Very High Positive
5	50.00-99.99	Very High Positive
6	>99.99	Very High Positive

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

5 to 9 days

**Performing Laboratory Location**

Eurofins Viracor

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

86331

86001

86003

82785

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FABP2	Allergic Bronchopulmonary Panel II	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z3248	Total IgE	In Process
Z3249	Aspergillus fumigatus IgE	6025-1
Z3250	Class	102153-4
Z3251	Aspergillus fumigatus IgG	26954-8
Z3252	A. fumigatus Mix Gel Diffusion	23821-2