



Test Definition: FFTIO

Thiocyanate, Serum

Overview

Method Name

Spectrophotometry (SPEC)

NY State Available

Yes

Specimen

Specimen Type

Varies

Specimen Required

Submit only 1 of the following specimens:

Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 1 mL sodium heparin plasma refrigerated in plastic vial.

Serum

Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 1 mL of serum refrigerated in plastic vial.

Specimen Minimum Volume

0.50 mL Does not allow for repeat testing

Reject Due To

| | |
|-----------|---|
| Hemolysis | Mild OK; Gross Reject |
| Lipemia | Mild OK; Gross OK |
| Icterus | Mild OK; Gross OK |
| Other | Anticoagulants other than Sodium heparin, EDTA, plain red-top |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Varies | Refrigerated (preferred) | 14 days | |
| | Ambient | 72 hours | |
| | Frozen | 180 days | |

Clinical & Interpretive**Reference Values**

Toxic Thiocyanate concentrations: Greater than 10 mg/dL

Performance**PDF Report**

No

Day(s) Performed

Monday through Sunday

Report Available

1 to 5 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

84430

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|--------------------|--------------------|
| FFTIO | Thiocyanate, Serum | 3002-3 |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z3316 | Thiocyanate | 3002-3 |