



# Test Definition: FMMM

Mephedrone, MDPV and Methylone, Urine

## Overview

### Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FMMCF	Mephedrone and MDPV Confirm	No	No
FMEYL	Methylone Confirm	No	No

### Testing Algorithm

If the Mephedrone, MDPV, Methylone is positive, then the Mephedrone and MDPV Confirm (FMMCF) and Methylone Confirm (FMEYL) will be performed at an additional charge.

Positive results will reflex additional testing at an additional charge.

If Mephedrone and/or MDPV are positive it will reflex the Mephedrone and MDPV Confirm (FMMCF). If the Methylone is positive it will reflex the Methylone Confirm (FMEYL).

### Method Name

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS) - Screen and Confirmation; if applicable

### NY State Available

Yes

## Specimen

### Specimen Type

Urine

### Specimen Required

Container: Preservative-free plastic urine container

Specimen volume: 10 mL

### Collection instructions

1. collect a random urine specimen
2. No preservative

### Specimen Minimum Volume

1 mL

### Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

## Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	180 days	

## Clinical & Interpretive

### Reference Values

Drug	Units	Reference Range
Mephedrone		Negative
MDPV		Negative
Methylone		Negative

Qualitative analysis for Mephedrone, Methylenedioxypropylone (MDPV) and Methylone

Screening threshold: 1.0 ng/mL

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Sunday

### Report Available

5 to 9 days

### Performing Laboratory Location

Medtox Laboratories, Inc.

---

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

**CPT Code Information**

80307-single drug

80371-(if appropriate)

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FMMM	Mephedrone, MDPV, Methylone	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z3417	Mephedrone	Unable to Verify
Z3418	MDPV	73687-6
Z3419	Methylone	73686-8