



# Test Definition: FADDS

Antidepressant Drug Screen, Qualitative

## Overview

### Method Name

Immunoassay (IA)

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Varies

### Specimen Required

Submit only 1 of the following:

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 3 mL of plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 3 mL of serum refrigerated in a plastic vial,

#### Urine

Collect 3 mL random urine and send refrigerated in a preservative free plastic urine container.

### Specimen Minimum Volume

0.6 mL

### Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
---------------	-------------	------	-------------------

Varies	Refrigerated (preferred)	7 days	
	Ambient	72 hours	
	Frozen	180 days	

## Clinical & Interpretive

### Reference Values

Antidepressant screen includes the analysis for:

Amytriptyline, Clomipramine and Desmethylclomipramine, Cyclobenzaprine, Desipramine, Doxepin and Desmethyldoxepin, Fluoxetine and Norfluoxetine, Imipramine, Maprotiline, Nortriptyline, Paroxetine, Protriptyline, Sertraline and Desmethylsertraline, Trimipramine.

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Sunday

### Report Available

3 to 9 days

### Performing Laboratory Location

Medtox Laboratories, Inc.

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### CPT Code Information

80333

80337

80369

### LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
---------	-----------------	--------------------

FADDS	Antidepressant Drug Screen	Not Provided
-------	----------------------------	--------------

Result ID	Test Result Name	Result LOINC® Value
Z3422	Specimen Type	31208-2
Z3423	Antidepressant Screen	Not Provided
Z3424	Amitriptyline	Not Provided
Z3425	Nortriptyline	Not Provided
Z3426	Imipramine	Not Provided
Z3427	Desipramine	Not Provided
Z3428	Sertraline	Not Provided
Z3429	Desmethylsertraline	Not Provided
Z3430	Fluoxetine	Not Provided
Z3431	Norfluoxetine	Not Provided
Z3432	Clomipramine	Not Provided
Z3433	Desmethylclomipramine	Not Provided
Z3434	Doxepin	Not Provided
Z3435	Desmethyldoxepin	Not Provided
Z3436	Maprotiline	Not Provided
Z3438	Trimipramine	Not Provided
Z3439	Paroxetine	Not Provided
Z3440	Cyclobenzaprine	Not Provided
Z3437	Protriptyline	Not Provided