

# **Test Definition: FACEC**

Angiotensin Converting Enzyme, CSF

## **Overview**

#### **Method Name**

**Quantitative Spectrophotometry** 

#### **NY State Available**

Yes

## **Specimen**

# **Specimen Type**

**CSF** 

#### **Specimen Required**

Container/Tube: Sterile vial Specimen Volume: 1 mL Collection Instructions:

- 1. Transfer 1 mL of spinal fluid to a sterile vial.
- 2. Send frozen.

**Note**: Gadolinium contrast agents have been reported to inhibit ACE activity. Therefore, CSF containing gadolinium-based contrast agents should not be submitted to the laboratory for evaluation.

#### **Specimen Minimum Volume**

0.5 mL

#### **Reject Due To**

Hemolysis	Reject
Xanthochromic	Reject
specimens	
(yellow color)	
Specimens	Reject
containing	
gadolinium-bas	
ed contrast	
agents	

### **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
CSF	Frozen (preferred)	180 days	



# **Test Definition: FACEC**

Angiotensin Converting Enzyme, CSF

Refrigerated	7 days	
--------------	--------	--

#### Clinical & Interpretive

#### **Clinical Information**

For more information see https://ltd.aruplab.com/.

#### **Reference Values**

0.0-2.5 U/L

#### **Performance**

#### **PDF Report**

No

#### Day(s) Performed

Monday, Wednesday, Friday

#### Report Available

1 to 7 days

#### **Performing Laboratory Location**

**ARUP Laboratories** 

#### **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **Test Classification**

This test was developed and its performance characteristics determined by ARUP Laboratories. The U.S. Food and Drug Administration has not approved or cleared this test; however, FDA clearance or approval is not currently required for clinical use. The results are not intended to be used as the sole means for clinical diagnosis or patient management decisions.

#### **CPT Code Information**

82164

#### **LOINC®** Information



# **Test Definition: FACEC**

Angiotensin Converting Enzyme, CSF

Test ID	Test Order Name	Order LOINC® Value
FACEC	Angiotensin Convert Enzyme CSF	12480-0

Result ID	Test Result Name	Result LOINC® Value
FACEC	Angiotensin Convert Enzyme CSF	12480-0