



Test Definition: FCURV

Curvularia spicifera/Bipolaris IgE

Overview

Method Name

ImmunoCAP FEIA

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Collection Container/Tube:

Preferred: Red top

Acceptable: Serum gel

Submission Container/tube: Plastic vial

Specimen Volume: 0.5 mL Serum

Collection Instructions:

1. Centrifuge and aliquot 0.5 mL of serum into a plastic vial.
2. Send refrigerate.

Specimen Minimum Volume

See Specimen Required

Reject Due To

Gross lipemia	Reject
---------------	--------

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	

Clinical & Interpretive

Clinical Information

Refer to www.eurofins-viracor.com/test-menu

Reference Values

<0.35 kU/L

Interpretation

Class	IgE (kU/L)	Comment
0	<.10	Negative
0/1	0.10 – 0.34	Equivocal/Borderline
1	0.35 – 0.69	Low Positive
2	0.70 – 3.49	Moderate Positive
3	3.50 – 17.49	High Positive
4	17.50 – 49.99	Very High Positive
5	50.00 – 99.9	Very High Positive
6	>99.99	Very High Positive

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

3 to 6 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86003

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FCURV	Curvularia spicifera/Bipolaris IgE	42198-2

Result ID	Test Result Name	Result LOINC® Value
Z3813	Curvularia spicifera/Bipolaris IgE	42198-2
Z3814	CLASS	102267-2