



Test Definition: FIGE

Fig (*Ficus carica*) IgE

Overview

Method Name

ImmunoCAP FEIA

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

| | |
|------------|-----------------------|
| Hemolysis: | Mild OK; Gross reject |
| Thawing: | Warm OK; Cold OK |
| Lipemia: | Mild OK; Gross reject |
| Icterus: | Mild OK; Gross reject |
| Other: | NA |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum | Refrigerated (preferred) | 28 days | |
| | Ambient | 28 days | |
| | Frozen | 365 days | |

Clinical & Interpretive

Reference Values

<0.35 kU/L

Interpretation

| <u>Class</u> | <u>IgE (kU/L)</u> | <u>Comment</u> |
|--------------|-------------------|----------------------|
| 0 | <0.10 | Negative |
| 0/1 | 0.10 – 0.34 | Equivocal/Borderline |
| 1 | 0.35 – 0.69 | Low Positive |
| 2 | 0.70 – 3.49 | Moderate Positive |
| 3 | 3.50 – 17.49 | High Positive |
| 4 | 17.50 – 49.99 | Very High Positive |
| 5 | 50.00 – 99.99 | Very High Positive |
| 6 | >99.99 | Very High Positive |

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

1 to 4 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86003

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
| FIGE | Fig IgE | 11173-2 |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z3870 | Fig IgE | 11173-2 |
| Z3871 | CLASS | 102738-2 |