

# **Test Definition: FMPG**

Poultry and Meat Panel IgG

# **Overview**

#### **Method Name**

Enzyme Immunoassay (FEIA)

#### **NY State Available**

No

# **Specimen**

# **Specimen Type**

Serum

# **Specimen Required**

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1 mL of serum refrigerated in a plastic vial.

#### **Specimen Minimum Volume**

1 mL

# **Reject Due To**

Hemolysis:	Mild OK; Gross reject
Thawing:	Warm OK; Cold OK
Lipemia:	Mild OK; Gross reject
Icterus:	Mild OK; Gross reject
Other:	NA

# **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	7 days	
	Frozen	365 days	

# **Clinical & Interpretive**

#### **Reference Values**

Reference ranges have not been established for food-specific IgG tests. The clinical utility of food-specific IgG tests has not been established. These tests can be used in special clinical situations to select foods for evaluation by diet



# **Test Definition: FMPG**

Poultry and Meat Panel IgG

elimination and challenge in patients who have food-related complaints. It should be recognized that the presence of food-specific IgG alone cannot be taken as evidence of food allergy and only indicates immunologic sensitization by the food allergen in question. This test should only be ordered by physicians who recognize the limitations of the test.

#### **Performance**

#### **PDF Report**

No

# Day(s) Performed

Monday through Friday

#### Report Available

3 to 7 days

#### **Performing Laboratory Location**

**Eurofins Viracor** 

#### **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

### **Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

#### **CPT Code Information**

86001 x 7

#### **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FMPG	Poultry and Meat Panel IgG	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z3961	Beef IgG	35529-7
Z3962	Chicken IgG	35532-1
Z3963	Egg White IgG	35535-4



# **Test Definition: FMPG**

Poultry and Meat Panel IgG

Z3964	Egg Yolk IgG	35536-2
Z3965	Lamb-Mutton IgG	60380-3
Z3966	Pork IgG	35543-8
Z3967	Turkey IgG	60411-6