



# Test Definition: FFPG

Fruit Panel IgG

## Overview

### Method Name

Enzyme immunoassay (FEIA)

### NY State Available

No

## Specimen

### Specimen Type

Serum

### Specimen Required

Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1.5 mL of serum refrigerated in a plastic vial.

### Specimen Minimum Volume

1.5 mL

### Reject Due To

Hemolysis	Mild OK; Gross reject
Lipemia	Mild OK; Gross reject
Icterus	Mild OK; Gross reject
Other	NA

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	7 days	
	Frozen	365 days	

## Clinical & Interpretive

### Reference Values

Reference ranges have not been established for food-specific IgG tests. The clinical utility of food-specific IgG tests has not been established. These tests can be used in special clinical situations to select foods for evaluation by diet

elimination and challenge in patients who have food-related complaints. It should be recognized that the presence of food-specific IgG alone cannot be taken as evidence of food allergy and only indicates immunologic sensitization by the food allergen in question. This test should only be ordered by physicians who recognize the limitations of the test.

## Performance

### PDF Report

No

### Day(s) Performed

Monday through Friday

### Report Available

3 to 7 days

### Performing Laboratory Location

Eurofins Viracor

## Fees & Codes

### Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

### Test Classification

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

### CPT Code Information

86001 x16

### LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FFPG	Fruit Panel IgG	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z3968	Apple IgG	60349-8
Z3969	Apricot IgG	60350-6
Z3970	Banana IgG	60352-2
Z3971	Blueberry IgG	58035-7

Z3972	Cranberry IgG	63104-4
Z3973	Grape IgG	60376-1
Z3974	Grapefruit IgG	60377-9
Z3975	Lemon IgG	60381-1
Z3976	Orange IgG	51904-1
Z3977	Papaya IgG	63131-7
Z3978	Peach IgG	60390-2
Z3979	Pear IgG	60391-0
Z3980	Pineapple IgG	60394-4
Z3981	Plum IgG	60395-1
Z3982	Raspberry IgG	63083-0
Z3983	Strawberry IgG	40835-1