



Test Definition: FRSE

Red Snapper (Lutjanus spp) IgE

Overview

Method Name

ImmunoCAP FEIA

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Collection Container/Tube:

Preferred: Red top

Acceptable: Serum gel

Submission Container/tube: Plastic vial

Specimen Volume: 0.5 mL Serum

Collection Instructions:

1. Centrifuge and aliquot 0.5 mL of serum into a plastic vial.
2. Send refrigerated.

Specimen Minimum Volume

See Specimen Required

Reject Due To

| | |
|---------|--------|
| Lipemia | Reject |
|---------|--------|

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|---------|-------------------|
| Serum | Refrigerated (preferred) | 28 days | |
| | Ambient | 28 days | |
| | Frozen | 28 days | |

Clinical & Interpretive

Clinical Information

Refer to www.eurofins-viracor.com/test-menu

Reference Values

<0.35 kU/L

Interpretation

| Class | IgE (kU/L) | Comment |
|-------|-------------|----------------------|
| 0 | <0.10 | Negative |
| 0/1 | 0.10-0.34 | Equivocal/Borderline |
| 1 | 0.35-0.69 | Low Positive |
| 2 | 0.70-3.49 | Moderate Positive |
| 3 | 3.50-17.49 | High Positive |
| 4 | 17.50-49.99 | Very High Positive |
| 5 | 50.00-99.99 | Very High Positive |
| 6 | >99.99 | Very High Positive |

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

3 to 6 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

[This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

CPT Code Information

86003

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
| FRSE | Red Snapper IgE | 7661-2 |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z3996 | Red Snapper IgE | 7661-2 |
| Z3997 | CLASS | 102601-2 |