

# **Test Definition: FSHAG**

Hickory Shagbark (Carya ovata) IgE

# **Overview**

#### **Method Name**

Enzyme Immunoassay (EIA)

#### **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Serum

# **Specimen Required**

Collection Container/Tube: Red top

Acceptable: Serum gel

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5

mL of serum refrigerated in a plastic vial.

#### **Specimen Minimum Volume**

0.5 mL

# **Reject Due To**

| Lipemia | Reject |
|---------|--------|

# **Specimen Stability Information**

| Specimen Type | Temperature              | Time     | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum         | Refrigerated (preferred) | 28 days  |                   |
|               | Ambient                  | 28 days  |                   |
|               | Frozen                   | 365 days |                   |

# **Clinical & Interpretive**

#### **Reference Values**

<0.35 kU/L



# **Test Definition: FSHAG**

Hickory Shagbark (Carya ovata) IgE

#### Interpretation

| Class | IgE (kU/L)  | <u>Comment</u>           |
|-------|-------------|--------------------------|
| 0     | <0.35       | <b>Below Detection</b>   |
| 1     | 0.35-0.69   | Low Positive             |
| 2     | 0.70-3.49   | <b>Moderate Positive</b> |
| 3     | 3.50-17.49  | Positive                 |
| 4     | 17.50-49.99 | Strong Positive          |
| 5     | >=49.99     | Very Strong Positive     |

# **Performance**

# **PDF Report**

No

# Day(s) Performed

Monday through Friday

# **Report Available**

4 to 7 days

# **Performing Laboratory Location**

**Eurofins Viracor** 

# **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

# **Test Classification**

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

#### **CPT Code Information**

86003

# **LOINC®** Information

| Test ID | Test Order Name                    | Order LOINC® Value |
|---------|------------------------------------|--------------------|
| FSHAG   | Hickory Shagbark (Carya ovata) IgE | 7404-7             |



# **Test Definition: FSHAG**

Hickory Shagbark (Carya ovata) IgE

| Result ID | Test Result Name                   | Result LOINC® Value |
|-----------|------------------------------------|---------------------|
| Z4018     | Hickory Shagbark (Carya ovata) IgE | 7404-7              |
| Z4019     | CLASS                              | 102754-9            |