



# Test Definition: FDLS

Donath Landsteiner

## Overview

### Method Name

Hemolysis

### NY State Available

Yes

## Specimen

### Specimen Type

Serum Red

### Specimen Required

**Collection Container/Tube:** Red top (serum gel/SST are **not acceptable**)

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 3 mL serum

#### Collection Instructions:

1. Maintain specimen at 37 degrees C until after centrifugation.
2. Centrifuge and aliquot 3 mL of serum into a plastic vial.
3. Send refrigerated.

### Specimen Minimum Volume

Serum: 2 mL

### Reject Due To

Gross hemolysis	Reject
Gross lipemia	Reject
Gross icterus	Reject

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Ambient	72 hours	

## Clinical & Interpretive

---

**Clinical Information**Refer to <https://ltd.aruplab.com/>**Reference Values**

Negative

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

1 to 7 days

**Performing Laboratory Location**

ARUP Laboratories

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**CPT Code Information**

86940

86941

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FDLS	Donath Landsteiner	18287-3

Result ID	Test Result Name	Result LOINC® Value
FDLS	Donath Landsteiner	18287-3