



Test Definition: XSRM

Additional Sample for Reflex Oligoclonal Banding, Serum

Overview

Method Name

Only orderable as part of a profile. For more information see MSP3 / Multiple Sclerosis (MS) Cascade, Serum and Spinal Fluid.

Collection Only

NY State Available

No

Specimen

Specimen Type

Serum

Specimen Required

Only orderable as part of a profile. For more information see MSP3 / Multiple Sclerosis (MS) Cascade, Serum and Spinal Fluid.

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Within 2 hours of collection, centrifuge and aliquot serum in plastic vial.
2. Label specimen as serum.

Specimen Minimum Volume

0.5 mL

Reject Due To

Gross hemolysis	OK
Gross lipemia	OK
Gross icterus	OK

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	14 days	
	Ambient	14 days	
	Refrigerated	14 days	

Clinical & Interpretive

Performance

PDF Report

No

Specimen Retention Time

14 days

Performing Laboratory Location

Mayo Clinic Laboratories - Rochester Superior Drive

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

Not Applicable

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
XSRM	Additional sample for reflex OLIGS	No LOINC Needed

Result ID	Test Result Name	Result LOINC® Value
XSRM	Additional sample for Reflex OLIGS	No LOINC Needed