



Test Definition: OXYMU

Oxymorphone Confirmation, Random, Urine

Overview

Useful For

Detection and quantification of oxymorphone and noroxymorphone in urine

Special Instructions

- [Clinical Toxicology CPT Code Client Guidance](#)

Method Name

Liquid Chromatography Tandem Mass Spectrometry (LC-MS/MS)

NY State Available

Yes

Specimen

Specimen Type

Urine

Ordering Guidance

For situations where chain of custody is required, a Chain of custody kit (T282) is available. For chain-of-custody testing, order OXYCX / Oxycodone with Metabolite Confirmation, Chain of Custody, Random, Urine.

Additional drug panels and specific requests are available; call 800-533-1710 or 507-266-5700.

Additional Testing Requirements

If urine creatinine is required or adulteration of the specimen is suspected, the following test should also be ordered ADULT / Adulterants Survey, Random, Urine.

Specimen Required

Supplies: Sarstedt Aliquot Tube 5 mL (T914)

Collection Container/Tube: Plastic urine container

Submission Container/Tube: Plastic, 5-mL tube

Specimen Volume: 3 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

Additional Information:

1. No specimen substitutions.
2. STAT requests are **not accepted** for this test.
3. Submitting less than 1 mL will compromise our ability to perform all necessary testing.

Forms

If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Minimum Volume

2.5 mL

Reject Due To

| | |
|-----------------|----|
| Gross hemolysis | OK |
| Gross icterus | OK |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|---------|-------------------|
| Urine | Refrigerated (preferred) | 28 days | |
| | Ambient | 14 days | |
| | Frozen | 28 days | |

Clinical & Interpretive

Clinical Information

Oxymorphone is metabolized in the liver to noroxymorphone and excreted via the kidney primarily as the glucuronide conjugates. Oxymorphone is also a metabolite of oxycodone and, therefore, the presence of oxymorphone could also indicate exposure to oxycodone.

The detection interval for opiates is generally 2 to 3 days after last ingestion.

Reference Values

Negative

Cutoff concentrations by liquid chromatography tandem mass spectrometry:

Oxymorphone: 25 ng/mL

Noroxymorphone: 25 ng/mL

Interpretation

This procedure reports the total urine concentration; this is the sum of the unconjugated and conjugated forms of the parent drug.

Cautions

Other drugs in the opioid class, such as fentanyl, meperidine, methadone, and opiate antagonists such as naloxone, are not detected.

Clinical Reference

1. Jutkiewicz EM, Traynor JR. Opioid analgesics. In: Brunton LL, Knollmann BC, eds. Goodman and Gilman's: The Pharmacological Basis of Therapeutics. 14th ed. McGraw-Hill Companies, Inc; 2023:chap 23
2. Baselt, RC. Disposition of Toxic Drugs and Chemical in Man. 10th ed. Biomedical Publications; 2014
3. Hackett LP, Dusci LJ, Ilett KF, Chiswell GM. Optimizing the hydrolysis of codeine and morphine glucuronides in urine. Ther Drug Monit. 2002;24(5):652-657
4. Langman LJ, Bechtel LK, Holstege CP. Clinical toxicology. In: Rifai N, Chiu RWK, Young I, Burnham CAD, Wittwer CT, eds. Tietz Textbook of Laboratory Medicine. 7th ed. Elsevier; 2023:chap 43

Performance

Method Description

Confirmation with quantification by liquid chromatography/mass spectrometry.(Unpublished Mayo method)

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

2 to 5 days

Specimen Retention Time

14 days

Performing Laboratory Location

Mayo Clinic Laboratories - Rochester Superior Drive

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Mayo Clinic in a manner consistent with CLIA requirements. It has not been cleared or approved by the US Food and Drug Administration.

CPT Code Information

G0480

80365 (if appropriate for select payers)

[Clinical Toxicology CPT Code Client Guidance](#)

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------------------|--------------------|
| OXYMU | Oxymorphone Confirmation, U | 94305-0 |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|----------------------------|---------------------|
| 62622 | Oxymorphone-by LC-MS/MS | 17395-5 |
| 36024 | Oxymorphone Interpretation | 69050-3 |
| 42014 | Noroxymorphone-by LC-MS/MS | 90894-7 |