

PAX8 Immunostain, Technical Component Only

Overview

Useful For

Aids in the identification of renal cell carcinomas, as well as papillary thyroid carcinomas and tumors of Mullerian origin

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
IHTOI	IHC Initial, Tech Only	No	No
IHTOA	IHC Additional, Tech Only	No	No

Testing Algorithm

For the initial technical component only immunohistochemical (IHC) stain performed, the appropriate bill-only test ID will be reflexed and charged (IHTOI). For each additional technical component only IHC stain performed, an additional bill-only test ID will be reflexed and charged (IHTOA).

Method Name

Immunohistochemistry (IHC)

NY State Available

Yes

Specimen

Specimen Type

TECHONLY

Ordering Guidance

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

Shipping Instructions

Attach the green "Attention Pathology" address label (T498) and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

Specimen Required

Specimen Type: Tissue

Supplies: Immunostain Technical Only Envelope (T693) **Container/Tube:** Immunostain Technical Only Envelope

Preferred:

-Formalin-fixed, paraffin-embedded tissue block



PAX8 Immunostain, Technical Component Only

OR

-2 Unstained, positively charged glass slides (25- x 75- x 1-mm) per test ordered; sections 4-microns thick **Acceptable**: None

Digital Image Access

- 1. Information on accessing digital images of immunohistochemical (IHC) stains and the manual requisition form can be accessed through this website: https://news.mayocliniclabs.com/pathology/digital-imaging/
- 2. Clients ordering stains using a manual requisition form will not have access to digital images.
- 3. Clients wishing to access digital images must place the order for IHC stains electronically. Information regarding digital imaging can be accessed through this website: https://news.mayocliniclabs.com/pathology/digital-imaging/#section3

Forms

If not ordering electronically, complete, print, and send a <u>Immunohistochemical (IHC)/In Situ Hybridization (ISH) Stains</u>
<u>Request</u> (T763) with the specimen.

Reject Due To

Wet/frozen	Reject
tissue	
Cytology	
smears	
Nonformalin	
fixed tissue	
Nonparaffin	
embedded	
tissue	
Noncharged	
slides	
ProbeOn slides	
Snowcoat	
slides	

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

Clinical & Interpretive

Clinical Information

PAX8 is a member of the paired box gene (PAX) family of transcription factors involved in kidney cell and thyroid development. PAX8 has been shown to be expressed in a high percentage of renal neoplasms, including both malignant renal cell carcinomas and benign renal tumors (oncocytomas). PAX8 has also been reported to be expressed in ovarian



PAX8 Immunostain, Technical Component Only

carcinomas.

Interpretation

This test does not include pathologist interpretation, only technical performance of the stain. If interpretation is required, order PATHC / Pathology Consultation for a full diagnostic evaluation or second opinion of the case.

The positive and negative controls are verified as showing appropriate immunoreactivity.

Interpretation of this test should be performed in the context of the patient's clinical history and other diagnostic tests by a qualified pathologist.

Cautions

Age of a cut paraffin section can affect immunoreactivity. Stability thresholds vary widely among published literature and are antigen dependent. Best practice is for paraffin sections to be cut within 6 weeks.

The charge of glass slides can be affected by environmental factors and subsequently may alter slide staining. Sending unsuitable glass slides can result in inconsistent staining due to poor slide surface chemistry.

Best practices for storage of positively charged slides:

- -Minimize time slides are stored after being unpackaged
- -Limit exposure to high humidity and heat
- -Minimize exposure to plastics

Clinical Reference

- 1. Chang A, Brimo F, Montgomery EA, Epstein JI. Use of PAX8 and GATA3 in diagnosing sarcomatoid renal cell carcinoma and sarcomatoid urothelial carcinoma. Hum Pathol. 2013;44(8):1563-1568
- 2. Hu Y, Hartmann A, Stoehr C, et al. PAX8 is expressed in the majority of renal epithelial neoplasms: an immunohistochemical study of 223 cases using a mouse monoclonal antibody. J Clin Pathol. 2012;65(3):254-256
- 3. Laury AR, Perets R, Piao H, et al: A comprehensive analysis of PAX8 expression in human epithelial tumors. Am J Surg Pathol. 2011;35(6):816-826
- 4. Ozcan A, Shen SS, Hamilton C, et al. PAX8 expression in non-neoplastic tissues, primary tumors, and metastatic tumors: a comprehensive immunohistochemical study. Mod Pathol. 2011;24(6):751-764
- 5. Tacha D, Qi W, Zhou D, Bremer R, Cheng L. PAX8 mouse monoclonal antibody (BC12) recognizes a restricted epitope and is highly sensitive in renal cell and ovarian cancers but does not cross-react with B cells and tumors of pancreatic origin. Appl Immunohistochem Mol Morphol. 2013;21(1):59-63
- 6. Tacha D, Zhou D, Cheng L. Expression of PAX8 in normal and neoplastic tissues: a comprehensive immunohistochemical study. Appl Immunohistochem Mol Morphol. 2011;19(4):293-299
- 7. Kilgore MR, Bosch DE, Adamson KH, Swanson PE, Dintzis SM, Rendi MH. Unexpected PAX8 immunoreactivity in metastatic high-grade breast cancer. Appl Immunohistochem Mol Morphol. 2019;27(9):637-643. doi:10.1097/PAI.00000000000000707
- 8. Magaki S, Hojat SA, Wei B, So A, Yong WH. An Introduction to the Performance of Immunohistochemistry. Methods Mol Biol. 2019;1897:289-298. doi:10.1007/978-1-4939-8935-5_25

Performance



PAX8 Immunostain, Technical Component Only

Method Description

Immunohistochemistry on sections of paraffin-embedded tissue.(Unpublished Mayo method)

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

1 to 3 days

Specimen Retention Time

Until staining is complete.

Performing Laboratory Location

Mayo Clinic Laboratories - Rochester Main Campus

Fees & Codes

Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

Test Classification

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

CPT Code Information

88342-TC, Primary 88341-TC, If additional IHC

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
PAX8	PAX8 IHC, Tech Only	Order only;no result

Result ID	Test Result Name	Result LOINC® Value
70846	PAX8 IHC, Tech Only	Bill only; no result