



# Test Definition: FD10S

Drugs of Abuse Screen, Serum

## Overview

## Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FAMCS	Amphetamines Confirmation, S	No	No
FBCS	Benzodiazepines Confirmation, S	No	No
FCANS	Cannabinoids Confirmation, S	No	No
FCMC	Cocaine and Metabolites Confirm, S	No	No
FMMCS	Methadone and Metabolite Confirm, S	No	No
FF10S	Fentanyl/Acetyl Fentanyl Confirm, S	No	No
FOFU	Opiates-Free Unconjugated Conf, S	No	No
FPCCS	Phencyclidine Confirmation, S	No	No
FBCFS	Barbiturates Confirmation, S	No	No

## Testing Algorithm

If appropriate, confirmation testing will be performed at an additional charge.

## Method Name

Enzyme-Linked Immunosorbent Assay (ELISA)

## NY State Available

Yes

## Specimen

### Specimen Type

Serum Red

### Specimen Required

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

**Specimen Type:** Serum

**Collection Container/Tube:** Red top

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 4 mL

**Collection Instructions:**

1. Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable.**
2. Centrifuge and send 4 mL of serum refrigerated in plastic, preservative-free vial.

**Specimen Minimum Volume**

1.75 mL

**Reject Due To**

Other	Polymer gel separation tube (SST or PST).
-------	---

**Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	14 days	

**Clinical & Interpretive**

**Reference Values**

Reporting limit determined each analysis.

<u>Test</u>	<u>Result</u>	<u>Units</u>
Opiates	None Detected	ng/mL
Cocaine/Metabolites	None Detected	ng/mL
Benzodiazepines	None Detected	ng/mL
Cannabinoids	None Detected	ng/mL
Amphetamines	None Detected	ng/mL
Barbiturates	None Detected	mcg/mL
Fentanyl/Acetyl Fentanyl	None Detected	ng/mL
Methadone/Metabolite	None Detected	ng/mL
Phencyclidine	None Detected	ng/mL
Methamphetamine/MDMA	None Detected	ng/mL
Oxycodone/Oxymorphone	None Detected	ng/mL

**Performance**

**PDF Report**

No

**Day(s) Performed**

Monday through Sunday

**Report Available**

5 to 14 days

**Performing Laboratory Location**

NMS Labs

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

**CPT Code Information**

80307

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FD10S	Drugs of Abuse Screen, S	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z5245	Opiates	8219-8
Z5246	Cocaine/Metabolites	73823-7
Z5247	Benzodiazepines	46976-7
Z5248	Cannabinoids	8172-9
Z5249	Amphetamines	8149-7
Z5250	Barbiturates	20421-4
Z5251	Methadone/Metabolite	91033-1
Z5252	Phencyclidine	8236-2
Z5253	Methamphetamine/MDMA	91034-9
Z5254	Oxycodone/Oxymorphone	77732-6
Z6248	Fentanyl/Acetyl Fentanyl	29356-3