

Overview

Reflex Tests

| Test Id | Reporting Name                      | Available Separately | Always Performed |
|---------|-------------------------------------|----------------------|------------------|
| FAMCS   | Amphetamines Confirmation, S        | No                   | No               |
| FBCS    | Benzodiazepines Confirmation, S     | No                   | No               |
| FCANS   | Cannabinoids Confirmation, S        | No                   | No               |
| FCMC    | Cocaine and Metabolites Confirm, S  | No                   | No               |
| FMMCS   | Methadone and Metabolite Confirm, S | No                   | No               |
| FF10S   | Fentanyl/Acetyl Fentanyl Confirm, S | No                   | No               |
| FOFU    | Opiates-Free Unconjugated Conf, S   | No                   | No               |
| FPCCS   | Phencyclidine Confirmation, S       | No                   | No               |
| FBCFS   | Barbiturates Confirmation, S        | No                   | No               |

Testing Algorithm

If appropriate, confirmation testing will be performed at an additional charge.

Method Name

Enzyme-Linked Immunosorbent Assay (ELISA)

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

**Specimen Type:** Serum  
**Collection Container/Tube:** Red top  
**Submission Container/Tube:** Plastic vial  
**Specimen Volume:** 4 mL  
**Collection Instructions:**  
1. Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable.**  
2. Centrifuge and send 4 mL of serum refrigerated in plastic, preservative-free vial.

**Specimen Minimum Volume**  
1.75 mL

**Reject Due To**

|       |   |
|-------|---|
| Other | Polymer gel separation tube (SST or PST). |
|-------|---|

**Specimen Stability Information**

| Specimen Type | Temperature              | Time    | Special Container |
|---------------|--------------------------|---------|-------------------|
| Serum Red     | Refrigerated (preferred) | 7 days  |                   |
|               | Frozen                   | 14 days |                   |

**Clinical & Interpretive**

**Reference Values**  
Reporting limit determined each analysis.

| Test                     | Result        | Units  |
|--------------------------|---------------|--------|
| Opiates                  | None Detected | ng/mL  |
| Cocaine/Metabolites      | None Detected | ng/mL  |
| Benzodiazepines          | None Detected | ng/mL  |
| Cannabinoids             | None Detected | ng/mL  |
| Amphetamines             | None Detected | ng/mL  |
| Barbiturates             | None Detected | mcg/mL |
| Fentanyl/Acetyl Fentanyl | None Detected | ng/mL  |
| Methadone/Metabolite     | None Detected | ng/mL  |
| Phencyclidine            | None Detected | ng/mL  |
| Methamphetamine/MDMA     | None Detected | ng/mL  |
| Oxycodone/Oxymorphone    | None Detected | ng/mL  |

**Performance**

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 14 days

Performing Laboratory Location

NMS Labs

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

80307

LOINC® Information

| Test ID | Test Order Name          | Order LOINC® Value |
|---------|--------------------------|--------------------|
| FD10S   | Drugs of Abuse Screen, S | Not Provided       |

| Result ID | Test Result Name         | Result LOINC® Value |
|-----------|--------------------------|---------------------|
| Z5245     | Opiates                  | 8219-8              |
| Z5246     | Cocaine/Metabolites      | 73823-7             |
| Z5247     | Benzodiazepines          | 46976-7             |
| Z5248     | Cannabinoids             | 8172-9              |
| Z5249     | Amphetamines             | 8149-7              |
| Z5250     | Barbiturates             | 20421-4             |
| Z5251     | Methadone/Metabolite     | 91033-1             |
| Z5252     | Phencyclidine            | 8236-2              |
| Z5253     | Methamphetamine/MDMA     | 91034-9             |
| Z5254     | Oxycodone/Oxymorphone    | 77732-6             |
| Z6248     | Fentanyl/Acetyl Fentanyl | 29356-3             |