



Test Definition: FDA1S

Drugs of Abuse (10 panel) and Alcohol Screen,
Serum

Overview

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FAMCS	Amphetamines Confirmation, S	No	No
FBCS	Benzodiazepines Confirmation, S	No	No
FCANS	Cannabinoids Confirmation, S	No	No
FCCM	Cocaine and Metabolites Confirm, S	No	No
FETCS	Ethanol Confirmation, S	No	No
FMMCS	Methadone and Metabolite Confirm, S	No	No
FOFU	Opiates-Free Unconjugated Conf, S	No	No
FPCCS	Phencyclidine Confirmation, S	No	No
FBCFS	Barbiturates Confirmation, S	No	No

Testing Algorithm

If appropriate, confirmation testing will be performed at an additional charge.

Method Name

Enzyme-Linked Immunosorbent Assay (ELISA)

Headspace Gas Chromatography (GC)

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

*****Must submit one specimen per order. Specimens cannot be shared between multiple orders.*****

Specimen Type: Serum

Collection Container/Tube: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 4 mL

Collection Instructions:

1. Collect sample using alcohol-free skin preparation.
2. Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable.**
3. Centrifuge and send 4 mL of serum refrigerated in a plastic, preservative-free vial.

Specimen Minimum Volume

2.00 mL

Reject Due To

Other	Polymer gel separation tube (SST or PST).
-------	---

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Frozen	14 days	

Clinical & Interpretive

Reference Values

Reporting limit determined each analysis.

<u>Test</u>	<u>Result</u>	<u>Units</u>
Ethanol	None Detected	mg/dL
Opiates	None Detected	ng/mL
Cocaine/Metabolites	None Detected	ng/mL
Benzodiazepines	None Detected	ng/mL
Cannabinoids	None Detected	ng/mL
Amphetamines	None Detected	ng/mL
Barbiturates	None Detected	mcg/mL
Methadone/Metabolite	None Detected	ng/mL
Phencyclidine	None Detected	ng/mL
Methamphetamine/MDMA	None Detected	ng/mL
Oxycodone/Oxymorphone	None Detected	ng/mL

Performance**PDF Report**

No

Day(s) Performed

Monday through Sunday

Report Available

6 to 20 days

Performing Laboratory Location

NMS Labs

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

80307x2

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FDA1S	Drug of Abuse 10 and Alcohol Scrn,S	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z5535	Ethanol	20470-1
Z5536	Opiates	8219-8
Z5537	Cocaine/Metabolites	73823-7
Z5538	Benzodiazepines	46976-7
Z5539	Cannabinoids	8172-9
Z5540	Amphetamines	8149-7

Test Definition: FDA1S

Drugs of Abuse (10 panel) and Alcohol Screen,
Serum

Z5541	Barbiturates	20421-4
Z5542	Methadone/Metabolite	59705-4
Z5543	Phencyclidine	8236-2
Z5544	Methamphetamine/MDMA	91034-9
Z5545	Oxycodone/Oxymorphone	77732-6