

# **Test Definition: FWCR1**

Walnut Component rJug r 1

# **Overview**

#### **Method Name**

ImmunoCAP FEIA

#### **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Serum

## **Specimen Required**

Specimen Type: Serum
Container/Tube: Red or SST
Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 0.5 mL

of serum refrigerated in a plastic vial.

#### **Specimen Minimum Volume**

0.5 mL

#### **Reject Due To**

Gross	Mild OK; Gross Reject
hemolysis:	
Thawing:	Warm OK; Cold OK
Gross lipemia:	Reject
Gross icterus:	NA
Other:	NA

#### **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	365 days	

# **Clinical & Interpretive**



# **Test Definition: FWCR1**

Walnut Component rJug r 1

#### **Reference Values**

<0.10 kU/L

#### Interpretation

Class	IgE (kU/L)	Comment
0	<0.10	Negative
0/1	0.10 - 0.34	Equivocal/Borderline
1	0.35 - 0.69	Low Positive
2	0.70 - 3.49	Moderate Positive
3	3.50 - 17.49	High Positive
4	17.50 - 49.99	Very High Positive
5	50.00 - 99.99	Very High Positive
6	>99.99	Very High Positive

#### **Performance**

#### **Method Description**

The ImmunoCAP FEIA method uses as the solid phase a flexible, hydrophobic cellulosic polymer to which allergen has been covalently linked. The advantage of this system is that it has a very high antigen binding capacity when compared to other systems and it has minimal non-specific binding with high total IgE.

#### **PDF Report**

No

#### **Performing Laboratory Location**

**Eurofins Viracor** 

#### Fees & Codes

#### Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

### **Test Classification**

This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.

#### **CPT Code Information**



# **Test Definition: FWCR1**

Walnut Component rJug r 1

86008

# **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FWCR1	Walnut Component rJug r 1	81790-8

Result ID	Test Result Name	Result LOINC® Value
Z5654	Walnut Component rJug r 1	81790-8
Z5655	Class	103119-4