

Test Definition: FAPMA

Anti-PM/Scl-100 Ab

Overview

Method Name

ELISA

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Collection Container/Tube: 5 mL Red **Submission Container/Tube:** Plastic vial

Specimen Volume: 1 mL

Acceptable: SST

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube is acceptable. Spin down and send 1 mL of

serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.3 mL (volume does NOT allow for repeat testing)

Reject Due To

Gross	Reject
hemolysis:	
Thawing:	Warm OK; Cold OK
Gross lipemia:	Reject
Gross icterus	Reject
Other:	Anything other than serum; bacterial contamination

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Ambient	7 days	
	Frozen	60 days	



Test Definition: FAPMA

Anti-PM/Scl-100 Ab

Clinical & Interpretive

Clinical Information

The anti-PM/Scl-100 antibody is associated with younger age, calcinosis and has lower rates of gastrointestinal symptoms, ILD and pulmonary hypertension. There is also evidence of a possibly better survival compared to the presence of either anti-PM/Scl-75 or anti-Scl-70 antibodies.

Reference Values

Reference Range: <20

Interpretation:

Negative: <20 units

Weak Positive: 20-39 units Moderate Positive: 40-80 units Strong Positive: >80 units

Performance

PDF Report

No

Report Available

16-25 days

Performing Laboratory Location

Esoterix Endocrinology

Fees & Codes

Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

Test Classification

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

CPT Code Information

86235



Test Definition: FAPMA

Anti-PM/Scl-100 Ab

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FAPMA	Anti-PM/Scl-100 Ab	101221-0

Result ID	Test Result Name	Result LOINC® Value
FAPMA	Anti-PM/Scl-100 Ab	101221-0