



Test Definition: FAU3R

Anti-U3 RNP Antibodies (Fibrillarin)

Overview

Method Name

RIPA Gel Radiography

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Collection Container/Tube: Red top or serum gel

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL Serum

Collection Instructions:

1. Within 1 hour of collection, centrifuge and aliquot 1 mL of serum into a plastic vial.
2. Send refrigerated.

Specimen Minimum Volume

Serum: 0.3 mL Note: This volume does not allow for repeat testing.

Reject Due To

Gross hemolysis:	Reject
Thawing:	Cold OK; Warm OK
Lipemia:	Reject
Icterus	Reject
Bacterial contamination	Reject

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Ambient	7 days	
	Frozen	60 days	

Clinical & Interpretive**Clinical Information**Refer to www.labcorp.com/test-menu/**Reference Values**

Negative

Performance**PDF Report**

No

Report Available

16 to 25 days

Performing Laboratory Location

Esoterix Endocrinology

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

CPT Code Information

86235

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FAU3R	Anti-U3 RNP (Fibrillarin)	49963-2

Result ID	Test Result Name	Result LOINC® Value
FAU3R	Anti-U3 RNP (Fibrillarin)	49963-2