



Test Definition: FSCBA

Synthetic Cannabinoid Metabolites
Screen-Expanded, Urine

Overview

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FSCBB	Synthetic Cannabinoid Confirm, U	No	No

Testing Algorithm

If appropriate synthetic cannabinoid confirmation will be performed at no additional charge.

Method Name

High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

NY State Available

Yes

Specimen

Specimen Type

Urine

Specimen Required

Must submit one specimen per order. Specimens cannot be shared between multiple orders.

Container/Tube: Plastic, preservative-free urine container

Specimen Volume: 5 mL

Collection Instructions:

1. Collect 5 mL random urine specimen without preservative.
2. Send refrigerated in a plastic preservative-free urine container.

Specimen Minimum Volume

2.4 mL

Reject Due To

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	30 days	

	Ambient	30 days	
	Frozen	30 days	

Clinical & Interpretive

Clinical Information

Refer to www.nmslabs.com/test-catalog.

Reference Values

Reporting limit determined each analysis.
MDMB-4en-PINACA t-Butylacetic acid: None detected
MDMB-BUTINACA t-Butylacetic acid: None detected
5F-MDMB-PINACA t-Butylacetic acid: None detected

Performance

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

10 to 16 days

Performing Laboratory Location

NMS Labs

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

Test Definition: FSCBA

Synthetic Cannabinoid Metabolites
Screen-Expanded, Urine

80307

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FSCBA	Synthetic Cannabinoid Screen, Urine	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z6439	MDMB-4en-PINACA t-Butylacetic acid	Not Provided
Z6440	MDMB-BUTINACA t-Butylacetic acid	Not Provided
Z6441	5F-MDMB-PINACA t-Butylacetic acid	Not Provided