

# **Test Definition: PROLX**

Fluphenazine (Prolixin), Serum

## **Overview**

#### **Method Name**

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

#### **NY State Available**

Yes

## **Specimen**

#### **Specimen Type**

Serum Red

## **Specimen Required**

**Collection Container/Tube:** Red top **Submission Container/Tube:** Plastic vial

Specimen Volume: 3 mL

Collection Instructions: Draw blood in a plain, red-top tube(s), serum-gel tube is not acceptable. Centrifuge and send 3

mL of serum refrigerate in a plastic vial.

## **Specimen Minimum Volume**

0.5 mL

## **Reject Due To**

Hemolysis	Mild reject; Gross reject	
Lipemia	Mild reject; Gross reject	
Icterus	Mild reject; Gross reject	
Other	Specimens other than Serum collected in red top. Plasma or whole blood collected in Sodium	
	Heparin or EDTA. Serum gel tubes or Plasma gel are NOT acceptable.	

## **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	180 days	

## **Clinical & Interpretive**



# **Test Definition: PROLX**

Fluphenazine (Prolixin), Serum

#### **Reference Values**

Reference Range: 1.0 - 10 ng/mL

#### **Performance**

## **PDF Report**

No

## Day(s) Performed

Monday through Sunday

## **Report Available**

5 to 9 days

#### **Specimen Retention Time**

2 weeks

## **Performing Laboratory Location**

Medtox Laboratories, Inc.

## **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **CPT Code Information**

80342

## **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
PROLX	Fluphenazine (Prolixin)	3650-9

Result ID	Test Result Name	Result LOINC® Value
Z1122	Fluphenazine	3650-9