

Overview

Method Name
Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

NY State Available
Yes

Specimen

Specimen Type
Varies

Specimen Required
Submit only 1 of the following specimens:

Plasma:
Specimen Type: Plasma
Collection Container/Tube: Green top
Submission Container/Tube: Plastic vial
Specimen Volume: 2 mL
Collection Instructions: Draw blood in a green-top sodium heparin tube(s), **plasma gel tube is not acceptable.** Centrifuge and send 2 mL of plasma refrigerated in a plastic vial.

Serum:
Specimen Type: Serum
Collection Container/Tube: Red top
Submission Container/Tube: Plastic vial
Specimen Volume: 2 mL
Collection Instructions: Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable.** Centrifuge and send 2 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume
0.6 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	Plasma gel tube, Serum gel tube

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Ambient	72 hours	
	Frozen	180 days	

Clinical & Interpretive

Reference Values

Reference Range: 50.0 - 240.0 ng/mL

Performance

PDF Report

No

Day(s) Performed

Monday, Tuesday, Thursday, Friday

Report Available

5 to 13 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80346
G0480 (if appropriate)

LOINC® Information

Test Definition: LORAZ

Lorazepam (Ativan), Serum

Test ID	Test Order Name	Order LOINC® Value
LORAZ	Lorazepam (Ativan)	59703-9

Result ID	Test Result Name	Result LOINC® Value
Z1123	Lorazepam (Ativan)	59703-9