

Pediatric Allergy Screen <3 Years, Serum

## Overview

#### **Useful For**

Establishing a diagnosis of an allergy to egg whites, milk, wheat, soybeans, and house dust mites

Defining the allergen responsible for eliciting signs and symptoms

Identifying allergens:

- -Responsible for allergic response or anaphylactic episode
- -To confirm sensitization prior to beginning immunotherapy
- -To investigate the specificity of allergic reactions to insect venom allergens, drugs, or chemical allergens

This test is **not useful for** patients previously treated with immunotherapy to determine if residual clinical sensitivity exists, or in patients in whom the medical management does not depend upon identification of allergen specificity.

#### **Profile Information**

| Test Id | Reporting Name             | Available Separately | Always Performed |
|---------|----------------------------|----------------------|------------------|
| EGG     | Egg White, IgE             | Yes                  | Yes              |
| MILK    | Milk, IgE                  | Yes                  | Yes              |
| WHT     | Wheat, IgE                 | Yes                  | Yes              |
| SOY     | Soybean, IgE               | Yes                  | Yes              |
| DF      | House Dust Mites/D.F., IgE | Yes                  | Yes              |

#### **Special Instructions**

• Allergens - Immunoglobulin E (IgE) Antibodies

#### **Method Name**

Fluorescence Enzyme Immunoassay (FEIA)

## **NY State Available**

Yes

## Specimen

# **Specimen Type**

Serum

## **Ordering Guidance**

For a listing of allergens available for testing, see Allergens - Immunoglobulin E (IgE) Antibodies



Pediatric Allergy Screen <3 Years, Serum

Specimen Required

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

**Collection Container/Tube:** 

**Preferred:** Serum gel **Acceptable:** Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL

Collection Instructions: Centrifuge and aliquot serum into a plastic vial.

#### **Forms**

If not ordering electronically, complete, print, and send an Allergen Test Request (T236) with the specimen.

### **Specimen Minimum Volume**

See Specimen Required

## Reject Due To

| Gross         | OK |
|---------------|----|
| hemolysis     |    |
| Gross lipemia | ОК |

#### **Specimen Stability Information**

| Specimen Type | Temperature              | Time    | Special Container |
|---------------|--------------------------|---------|-------------------|
| Serum         | Refrigerated (preferred) | 14 days |                   |
|               | Frozen                   | 90 days |                   |

## Clinical & Interpretive

## **Clinical Information**

Clinical manifestations of immediate hypersensitivity (allergic) diseases are caused by the release of proinflammatory mediators (histamine, leukotrienes, and prostaglandins) from IgE-sensitized effector cells (mast cells and basophils) when cell-bound IgE antibodies interact with an allergen.

In vitro serum testing for IgE antibodies provides an indication of the immune response to allergens that may be associated with allergic disease.

The allergens chosen for testing often depend upon the age of the patient, history of allergen exposure, season of the year, and clinical manifestations. In individuals predisposed to develop allergic disease, the sequence of sensitization and clinical manifestations proceed as follows: eczema and respiratory disease (rhinitis and bronchospasm) in infants and children younger than 5 years due to food sensitivity (milk, egg, soy, and wheat proteins) followed by respiratory disease (rhinitis and asthma) in older children and adults due to sensitivity to inhalant allergens (dust mite, mold, and pollen inhalants).



Pediatric Allergy Screen <3 Years, Serum

#### **Reference Values**

| Class | lgE kU/L  | Interpretation       |
|-------|-----------|----------------------|
| 0     | <0.10     | Negative             |
| 0/1   | 0.10-0.34 | Borderline/equivocal |
| 1     | 0.35-0.69 | Equivocal            |
| 2     | 0.70-3.49 | Positive             |
| 3     | 3.50-17.4 | Positive             |
| 4     | 17.5-49.9 | Strongly positive    |
| 5     | 50.0-99.9 | Strongly positive    |
| 6     | > or =100 | Strongly positive    |

Concentrations of 0.70 kU/L or more (class 2 and above) will flag as abnormally high.

Reference values apply to all ages.

#### Interpretation

Detection of IgE antibodies in serum (class 1 or greater) indicates an increased likelihood of allergic disease as opposed to other etiologies and defines the allergens that may be responsible for eliciting signs and symptoms.

The level of IgE antibodies in serum varies directly with the concentration of IgE antibodies expressed as a class score or kU/L.

#### **Cautions**

Some individuals with clinically insignificant sensitivity to allergens may have measurable levels of IgE antibodies in serum, and test results must be interpreted in the clinical context.

False-positive results for IgE antibodies may occur in patients with markedly elevated serum IgE (>2500 kU/L) due to nonspecific binding to allergen solid phases.

#### **Clinical Reference**

Homburger HA, Hamilton RG. Allergic diseases. In: McPherson RA, Pincus MR, eds. Henry's Clinical Diagnosis and Management by Laboratory Methods. 23rd ed. Elsevier; 2017:1057-1070

#### **Performance**

### **Method Description**

Specific IgE from the patient's serum reacts with the allergen of interest, which is covalently coupled to an ImmunoCAP. After washing away nonspecific IgE, enzyme-labeled anti-IgE antibody is added to form a complex. After incubation, unbound anti-IgE is washed away, and the bound complex incubated with a developing agent. After stopping the reaction, the fluorescence of the eluate is measured. Fluorescence is proportional to the amount of specific IgE present in the patient's sample (ie, the higher the fluorescence value, the more IgE antibody is present). (Package insert: ImmunoCAP System Specific IgE FEIA. Phadia; Rev 06/2020)



Pediatric Allergy Screen <3 Years, Serum

## **PDF Report**

No

## Day(s) Performed

Monday through Friday

## **Report Available**

Same day/1 to 3 days

### **Specimen Retention Time**

14 days

## **Performing Laboratory Location**

Mayo Clinic Laboratories - Rochester Superior Drive

#### Fees & Codes

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

# **Test Classification**

This test has been cleared, approved, or is exempt by the US Food and Drug Administration and is used per manufacturer's instructions. Performance characteristics were verified by Mayo Clinic in a manner consistent with CLIA requirements.

#### **CPT Code Information**

86003 x 5

## **LOINC®** Information

| Test ID | Test Order Name                  | Order LOINC® Value |
|---------|----------------------------------|--------------------|
| PAS3    | Pediatric Allergy Scrn <3 Yrs, S | 94593-1            |

| Result ID | Test Result Name           | Result LOINC® Value |
|-----------|----------------------------|---------------------|
| DF        | House Dust Mites/D.F., IgE | 6095-4              |
| EGG       | Egg White, IgE             | 6106-9              |
| MILK      | Milk, IgE                  | 6174-7              |
| SOY       | Soybean, IgE               | 6248-9              |
| WHT       | Wheat, IgE                 | 6276-0              |