



# Test Definition: FFLUR

Flurazepam (Dalmane) and Desalkylflurazepam

## Overview

### Method Name

Liquid Chromatography/Tandem Mass Spectrometry (LC/MS/MS)

### NY State Available

Yes

## Specimen

### Specimen Type

Varies

### Specimen Required

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 2 mL sodium heparin plasma frozen in plastic vial.

#### Serum

Draw blood in a plain, red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 2 mL of serum frozen in plastic vial.

### Specimen Minimum Volume

0.3 mL

### Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

### Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Varies	Frozen	180 days	

## Clinical & Interpretive

**Reference Values**

Flurazepam:

Reference Range: 0 - 30 ng/mL

Desalkylflurazepam:

Reference Range: 30 - 150 ng/mL

Flurazepam + Desalkylflurazepam:

Reference Range: 30 - 180 ng/mL

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Sunday

**Report Available**

5 to 9 days

**Specimen Retention Time**

2 weeks

**Performing Laboratory Location**

Medtox Laboratories, Inc.

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**CPT Code Information**

80346

**LOINC® Information**

Test ID	Test Order Name	Order LOINC® Value
FFLUR	Flurazepam (Dalmane)	73826-0

## Test Definition: FFLUR

Flurazepam (Dalmane) and Desalkylflurazepam

Result ID	Test Result Name	Result LOINC® Value
Z1133	Flurazepam	46979-1
Z1160	Desalkylflurazepam	46980-9
Z1161	Flurazepam + Desalkylflurazepam	73826-0