

Estradiol Free, Serum (includes Estradiol and SHBG)

### **Overview**

#### **Method Name**

Liquid Chromatography Tandem Mass Spectrometry (LC-MS/MS); Equilibrium Dialysis; Immunoradiometric Assay (IRMA)

### **NY State Available**

Yes

## Specimen

## **Specimen Type**

Serum Red

### **Specimen Required**

**Collection Container/Tube:** 

Preferred: Red top

Acceptable: Serum gel tube

Submission Container/tube: Plastic vial

**Specimen Volume**: 3 mL **Collection Instructions**:

1. Within 45 minutes of collection, centrifuge and aliquot 3 mL of serum into a plastic vial.

2. Freeze immediately after separation and send frozen.

### **Specimen Minimum Volume**

1 mL (Note: This volume does not allow for repeat testing.)

## **Reject Due To**

Plasma	Reject

## **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum Red	Frozen (preferred)	180 days	
	Ambient	48 hours	
	Refrigerated	48 hours	

## **Clinical & Interpretive**



Estradiol Free, Serum (includes Estradiol and SHBG)

#### **Reference Values**

Estradiol, Serum MS Units: pg/mL

Newborn:

Levels are markedly elevated at birth and fall rapidly during the first week to prepubertal values of <15.

Males <6 m:

Levels increase to 10 - 32 between 30 and 60 days, then decline to prepubertal levels of <15 by six months.

Females <1 y:

Levels increase to 5.0 - 50 between 30 and 60 days, then decline to prepubertal levels of <15 during the first year.

Prepubertal: <15

Adult Males: 8.0 - 35

Adult Females: Follicular: 30 - 100 Luteal: 70 - 300 Postmenopausal: <15

Free Estradiol, Percent

Units: %

Adult Males: 1.7 - 5.4

Adult Females: 1.6 - 3.6

Free Estradiol, Serum

Units: pg/mL

Adult Males: 0.2 - 1.5

Adult Females: 0.6 - 7.1

Sex Hormone Binding Globulin

Units: nmol/L

Infants (1 - 23m): 60.0 - 252.0

Prepubertal: 72.0 - 220.0

Pubertal:



Estradiol Free, Serum (includes Estradiol and SHBG)

Males:16.0 - 100.0 Females: 36.0 - 125.0

Adult Males:

20 - 49 y: 16.5 - 55.9 >49y: 19.3 - 76.4

Adult Females:

20 - 49y: 24.6 - 122.0 >49y: 17.3 - 125.0

### **Performance**

### **PDF Report**

No

## Day(s) Performed

Varies

## Report Available

10 to 16 days

## **Performing Laboratory Location**

**Esoterix Endocrinology** 

### **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

### **Test Classification**

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

### **CPT Code Information**

82681 - Free Estradiol

84270 - Sex Hormone binding globulin

82670 - Estradiol



Estradiol Free, Serum (includes Estradiol and SHBG)

## **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FFES	Free Estradiol with SHBG	Not Provided

Result ID	Test Result Name	Result LOINC® Value
Z0228	Free Estradiol, Percent	14960-9
Z0229	Free Estradiol, Serum	2240-0
Z0230	Sex Hormone Binding Globulin (SHBG)	13967-5
Z0259	Estradiol, Serum	35384-7