
Reporting Title: MSUD Gene Panel

Performing Location: Mayo Clinic Laboratories - Rochester Main Campus

Ordering Guidance:

The recommended first-tier screening tests for maple syrup urine disease (MSUD) are a combination of biochemical tests including quantitative plasma amino acids and urine organic acids. Order AAQP / Amino Acids, Quantitative, Plasma and OAU / Organic Acids Screen, Random, Urine.

Customization of this panel and single gene analysis for any gene present on this panel is available. For more information see CGPH / Custom Gene Panel, Hereditary, Next-Generation Sequencing, Varies. To modify this panel via CGPH, please use the Inborn Errors of Metabolism disease state for step 1 on the [Custom Gene Ordering Tool](#).

Targeted testing for familial variants (also called site-specific or known mutations testing) is available for the genes on this panel. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

Shipping Instructions:

Specimen Requirements:

Patient Preparation: A previous hematopoietic stem cell transplant from an allogenic donor will interfere with testing. For information about testing patients who have received a hematopoietic stem cell transplant, call 800-533-1710.

Submit only 1 of the following specimens:

Specimen Type: Whole blood

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Green top (sodium heparin)

Specimen Volume: 3 mL

Collection Instructions:

1. Invert several times to mix blood.
2. Send whole blood specimen in original tube. **Do not aliquot.**
3. Whole blood collected postnatal from an umbilical cord is also acceptable. See Additional Information.

Specimen Stability Information: Ambient (preferred) 4 days/Refrigerated 4 days/Frozen 4 days

Additional Information:

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.
2. To ensure minimum volume and concentration of DNA are met, the requested volume must be submitted. Testing may be canceled if DNA requirements are inadequate.
3. For postnatal umbilical cord whole blood specimens, maternal cell contamination studies are recommended to ensure test results reflect that of the patient tested. A maternal blood specimen is required to complete maternal cell contamination studies. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on both the cord blood and maternal blood specimens under separate order numbers.

Specimen Type: Saliva

Patient Preparation: Patient should not eat, drink, smoke, or chew gum 30 minutes prior to collection.

Supplies:

DNA Saliva Kit High Yield (T1007)

Saliva Swab Collection Kit (T786)

Container/Tube:

Preferred: High-yield DNA saliva kit

Acceptable: Saliva swab

Specimen Volume: 1 Tube if using T1007 or 2 swabs if using T786

Collection Instructions: Collect and send specimen per kit instructions.

Specimen Stability Information: Ambient (preferred) 30 days/Refrigerated 30 days

Additional Information: Saliva specimens are acceptable but not recommended. Due to lower quantity/quality of DNA yielded from saliva, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

Specimen Type: Blood spot

Supplies: Card-Blood Spot Collection (Filter Paper) (T493)

Container/Tube:

Preferred: Collection card (Whatman Protein Saver 903 Paper)

Acceptable: PerkinElmer 226 filter paper or blood spot collection card

Specimen Volume: 2 to 5 Blood spots

Collection Instructions:

1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect a Dried Blood Spot Sample](#).
2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.
3. Do not expose specimen to heat or direct sunlight.
4. Do not stack wet specimens.
5. Keep specimen dry.

Specimen Stability Information: Ambient (preferred)/Refrigerated

Additional Information:

1. Blood spot specimens are acceptable but not recommended. Due to lower quantity/quality of DNA yielded from blood spots, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.
2. Due to lower concentration of DNA yielded from blood spot, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.
3. For collection instructions, see [Blood Spot Collection Instructions](#)
4. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)
5. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

Specimen Type: Skin biopsy

Supplies: Fibroblast Biopsy Transport Media (T115)

Container/Tube: Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.

Specimen Volume: 4-mm Punch

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Cultured fibroblasts

Source: Skin or tissue

Container/Tube: T-25 flask

Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a skin or tissue biopsy. Cultured cells from a prenatal specimen will not be accepted.

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Extracted DNA

Container/Tube:

Preferred: Screw Cap Micro Tube, 2 mL with skirted conical base

Acceptable: Matrix tube, 1 mL

Collection Instructions:

1. The preferred volume is at least 100 µL at a concentration of 75 ng/µL.
2. Include concentration and volume on tube.

Specimen Stability Information: Frozen (preferred) 1 year/Ambient/Refrigerated

Additional Information: DNA must be extracted in a CLIA-certified laboratory or equivalent and must be extracted from a specimen type listed as acceptable for this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

Prenatal Specimens

Due to its complexity, consultation with the laboratory is required for all prenatal testing; call 800-533-1710 to speak to a genetic counselor.

Specimen Type: Amniotic fluid

Container/Tube: Amniotic fluid container

Specimen Volume: 20 mL

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information: Specimen will only be tested after culture.

-
1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
 2. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid. An additional 2 to 3 weeks are required to culture amniotic fluid before genetic testing can occur.
 3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Confluent cultured amniocytes

This does not include cultured chorionic villi.

Container/Tube: T-25 flask

Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured amniocytes from another laboratory

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Chorionic villi

Container/Tube: 15-mL tube containing 15 mL of transport media

Specimen Volume: 20 mg

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information: Specimen will only be tested after culture.

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Cultured chorionic villi

Container/Tube: T-25 flasks

Specimen Volume: 2 Full flasks

Collection Instructions: Submit confluent cultured cells from another laboratory.

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Forms:

1. **New York Clients-Informed consent is required.** Document on the request form or electronic order that a copy is on file. The following documents are available in:

-[Informed Consent for Genetic Testing](#) (T576)

-[Informed Consent for Genetic Testing \(Spanish\)](#) (T826)

2. [Molecular Genetics: Biochemical Disorders Patient Information](#) (T527)

3. If not ordering electronically, complete, print, and send a [Biochemical Genetics Test Request](#) (T798) with the specimen.

Specimen Type	Temperature	Time	Special Container
Varies	Varies		

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
608776	Test Description	Alphanumeric		62364-5
608777	Specimen	Alphanumeric		31208-2
608778	Source	Alphanumeric		31208-2
608779	Result Summary	Alphanumeric		50397-9
608780	Result	Alphanumeric		82939-0
608781	Interpretation	Alphanumeric		69047-9
608782	Resources	Alphanumeric		99622-3
608783	Additional Information	Alphanumeric		48767-8
608784	Method	Alphanumeric		85069-3
608785	Genes Analyzed	Alphanumeric		48018-6
608786	Disclaimer	Alphanumeric		62364-5
608787	Released By	Alphanumeric		18771-6

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

Supplemental

CPT Code Information:

81405

81406 x 3

81479

88233-Tissue culture, skin, solid tissue biopsy (if appropriate)

88240-Cryopreservation (if appropriate)

81479 (if appropriate for government payers)

Reflex Tests:

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
---------	----------------	-----------	----------	------------------	----------------------

_STR1	Comp Analysis using STR (Bill only)	1	81265	No	No, (Bill only)
_STR2	Add'l comp analysis w/STR (Bill Only)	1	81266	No	No, (Bill only)
CULFB	Fibroblast Culture for Genetic Test	1	88233	No	Yes
CULAF	Amniotic Fluid Culture/Genetic Test	1	88235	No	Yes
MATCC	Maternal Cell Contamination, B	1	81265	No	Yes

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
CULAF	52304	Result Summary	Alphanumeric		50397-9
CULAF	52306	Interpretation	Alphanumeric		69965-2
CULAF	52305	Result	Alphanumeric		82939-0
CULAF	CG767	Reason for Referral	Alphanumeric		42349-1
CULAF	52307	Specimen	Alphanumeric		31208-2
CULAF	52308	Source	Alphanumeric		31208-2
CULAF	52309	Method	Alphanumeric		85069-3
CULAF	54641	Additional Information	Alphanumeric		48767-8
CULAF	52310	Released By	Alphanumeric		18771-6
CULFB	52327	Result Summary	Alphanumeric		50397-9
CULFB	52329	Interpretation	Alphanumeric		69965-2
CULFB	52328	Result	Alphanumeric		82939-0
CULFB	CG770	Reason for Referral	Alphanumeric		42349-1
CULFB	CG899	Specimen	Alphanumeric		31208-2
CULFB	52331	Source	Alphanumeric		31208-2
CULFB	52332	Method	Alphanumeric		85069-3
CULFB	54625	Additional Information	Alphanumeric		48767-8
CULFB	52333	Released By	Alphanumeric		18771-6
MATCC	53285	Result Summary	Alphanumeric		50397-9
MATCC	53286	Result	Alphanumeric		40704-9
MATCC	53287	Interpretation	Alphanumeric		69047-9
MATCC	53288	Reason for referral	Alphanumeric		42349-1
MATCC	53289	Specimen	Alphanumeric		31208-2
MATCC	53290	Source	Alphanumeric		31208-2
MATCC	53291	Released By	Alphanumeric		18771-6
MATCC	55150	Method	Alphanumeric		85069-3

Reference Values:

An interpretive report will be provided.