

Reporting Title: Drug of Abuse, Amphetamine, U
Performing Location: New England

Specimen Requirements:

Container/Tube: Plastic, 10-mL urine tube (Supply T068)

Specimen Volume: 10 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative.

Specimen Type	Temperature	Time
Urine	Refrigerated (preferred)	14 days
	Ambient	14 days
	Frozen	14 days

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
11164	Amphetamines Also used by tests: DAU9, DAU7, DAU8, DAU	Numeric	ng/mL	8150-5

CPT Code: 1 × 80101**Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
DAUAM	Drug of Abuse, Amphetamine Conf, U	2	82145	No	Yes

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
DAUAM	501207	GC/MS Confirmation - Amphetamine	Alphanumeric		16234-7
DAUAM	501208	Amphetamine	Alphanumeric	ng/mL	16234-7



Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
DAUAM	501209	Methamphetamine	Alphanumeric	ng/mL	16235-4
DAUAM	500197	MDMA (Ecstasy)	Alphanumeric	ng/mL	18358-2
DAUAM	501210	MDA (Ecstasy Metabolite)	Alphanumeric	ng/mL	27409-2
DAUAM	501211	Phentermine	Alphanumeric	ng/mL	20557-5
DAUAM	501212	Pseudoephedrine/Ephedrine	Alphanumeric	ng/mL	26785-6
DAUAM	501213	Phenylpropanolamine	Alphanumeric	ng/mL	17443-3
DAUAM	501214	Comment	Alphanumeric		In Process

Reference Values:

Negative

EMIT cutoff concentration: 500 ng/mL

Positives are reported with a quantitative GC/MS result.