



Reporting Title: Sesame Seed Component Reflex, S

Performing Location: Rochester

Ordering Guidance:

For a listing of allergens available for testing, see Allergens - Immunoglobulin E (IgE) Antibodies

Specimen Requirements:

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.6 mL

Collection Instructions: Centrifuge and aliquot serum into a plastic vial.

Specimen Minimum Volume:

0.4 mL

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	90 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
SESA1	Sesame Seed, IgE, S	Numeric	kU/L	6242-2

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code: 1 × 86003

**Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
SESPR	Sesame Seed Component, IgE, S	1	86008	No	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
SESPR	SESI1	Ses i 1, IgE, S	Numeric	kU/L	100350-8
SESPR	INSSC	Sesame Seed IgE Interpretation	Alphanumeric		69048-7

Reference Values:

Class	IgE kU/L	Interpretation
0	<0.10	Negative
0/1	0.10-0.34	Borderline/equivocal
1	0.35-0.69	Equivocal
2	0.70-3.49	Positive
3	3.50-17.4	Positive
4	17.5-49.9	Strongly positive
5	50.0-99.9	Strongly positive
6	> or =100	Strongly positive

Concentrations of 0.70 kU/L or more (class 2 and above) will flag as abnormally high.

Reference values apply to all ages.