

NEW REFERRAL TEST

Notification Date: May 20, 2022 Effective Date: June 06, 2022

Hypersensitivity Pneumonitis Panel

Explanation: The following new orderable, referred to Eurofins Viracor, will be available on 06/06/2022.

Test ID: FHSPP

Methods:

Immunoassay FEIA

Reference Values:

Alternaria tenuis/alternata IgG	<12.0 mcg/mL
Aspergillus fumigatus IgG	<46.0 mcg/mL
Aureobasidium pullulans IgG	<18.0 mcg/mL
Laceyella sacchari IgG	<25.0 mcg/mL
Micropolyspora faeni IgG	<5.0 mcg/mL
Penicillium Chrysogenum/notatum IgG	<22.0 mcg/mL
Phoma betae IgG	<8.0 mcg/mL
Trichoderma viride IgG	<10.0 mcg/mL

Specimen Requirements:

Container/Tube:	
Preferred:	Red top
Acceptable:	Serum gel
Specimen Volume:	0.6 mL
Collection Instructions:	Draw blood in a plain, red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.6 mL of serum refrigerated in a plastic vial.
Minimum Volume:	0.5 mL

Specimen Stability Information:

Specimen Type	Temperature	Time
Serum	Refrigerated (preferred)	28 days
	Ambient	7 days
	Frozen	28 days

Fee: \$153.40 **CPT Code:** 86001 x8

Day(s) Setup: Monday through Friday

Report Available: 2 to 7 days

Questions:

Contact MCL Referrals Supervisor Amy Bluhm at 800-533-1710.