

## 21-Hydroxylase Gene, CYP21A2, Full Gene Analysis, Varies

**Test ID:** CYPZ

**Explanation:** On the effective date, the Method, Specimen Required, and Specimen Stability will be updated for this assay.

### Current Method

Polymerase Chain Reaction (PCR) Amplification followed by DNA Sequence Analysis and Gene Dosage Analysis by Multiplex Ligation-Dependent Probe Amplification (MLPA)

### New Method

Polymerase Chain Reaction (PCR) Amplification followed by DNA Sequence Analysis and Gene Dosage Analysis by Multiplex Ligation-Dependent Probe Amplification (MLPA) and droplet digital PCR (ddPCR)

### Current Specimen Required

**Patient Preparation:** A previous bone marrow transplant from an allogenic donor will interfere with testing. For testing patients who have received a bone marrow transplant, call 800-533-1710 for instructions.

**Submit only 1 of the following specimens:**

**Specimen Type:** Whole blood

**Container/Tube:**

**Preferred:** Lavender top (EDTA)

**Specimen Volume:** 3 mL

**Collection Instructions:**

1. Invert several times to mix blood.
  2. Send specimen in original tube. **Do not** aliquot.
- Specimen Stability Information:** Ambient (preferred)  
4 days/Refrigerated 14 days

#### Prenatal Specimens

Due to its complexity, consultation with the laboratory is required for all prenatal testing; call 800-533-1710 to speak to a genetic counselor.

**Specimen Type:** Amniotic fluid

**Container/Tube:** Amniotic fluid container

**Specimen Volume:** 20 mL

**Specimen Stability Information:** Refrigerated (preferred)/Ambient

**Additional information:**

### New Specimen Required

**Patient Preparation:** A previous bone marrow transplant from an allogenic donor will interfere with testing. For information about testing patients who have received a bone marrow transplant, call 800-533-1710.

**Submit only 1 of the following specimens:**

**Specimen Type:** Whole blood

**Container/Tube:**

**Preferred:** Lavender top (EDTA) or yellow top (ACD)

**Acceptable:** None

**Specimen Volume:** 3 mL

**Collection Instructions:**

1. Invert several times to mix blood.
2. Send whole blood specimen in original tube. **Do not** aliquot.

**Specimen Stability Information:** Ambient 4 days (preferred)/Refrigerated 4 days/Frozen 4 days

#### Additional Information:

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.
2. To ensure minimum volume and concentration of DNA is met, the requested volume must be submitted. Testing may be canceled if DNA requirements are inadequate.

**Specimen Type:** Cord blood

**Container/Tube:**

1. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid. An additional 2 to 3 weeks is required to culture amniotic fluid before genetic testing can occur.
2. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

**Specimen Type: Chorionic villi**

**Container/Tube:** 15-mL tube containing 15 mL of transport media

**Specimen Volume:** 20 mg

**Specimen Stability Information:** Refrigerated

**Additional Information:**

1. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.
2. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

**Acceptable**

**Specimen Type:** Confluent cultured cells

**Container/Tube:** T-25 flask

**Specimen Volume:** 2 Flasks

**Collection Instructions:** Submit confluent cultured cells from another laboratory.

**Specimen Stability Information:** Ambient (preferred)/Refrigerated

**Additional Information:** All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

**Preferred:** Lavender top (EDTA) or yellow top (ACD)

**Acceptable:** None

**Specimen Volume:** 3 mL

**Collection Instructions:**

1. Invert several times to mix blood.
2. Send specimen in original tube. **Do not aliquot.**

**Specimen Stability Information:** Ambient (preferred) 4 days/Refrigerated 4 days/Frozen, 4 days

**Additional Information:**

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.
2. To ensure minimum volume and concentration of DNA is met, the preferred volume of blood must be submitted. Testing may be canceled if DNA requirements are inadequate.
3. While a properly collected cord blood sample may not be at risk for maternal cell contamination, unanticipated complications may occur during collection. Therefore, maternal cell contamination studies are recommended to ensure the test results reflect that of the patient tested and are available at an additional charge.
4. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

**Specimen Type: Extracted DNA**

**Container/Tube:**

**Preferred:** Screw Cap Micro Tube, 2 mL with skirted conical base

**Acceptable:** Matrix tube, 1 mL

**Collection Instructions:**

1. The preferred volume is at least 100 mcL at a concentration of 75 ng/mcL.
2. Include concentration and volume on tube.

**Specimen Stability Information:** Frozen (preferred) 1 year/Ambient/Refrigerated

**Additional Information:** DNA must be extracted in a CLIA-certified laboratory or equivalent and must be extracted from a specimen type listed as acceptable for this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

**Prenatal Specimens**

Due to its complexity, consultation with the laboratory is required for all prenatal testing; call 800-533-1710 to speak to a genetic counselor.

**Preferred:**

**Specimen Type: Amniotic fluid**

**Container/Tube:** Amniotic fluid container

**Specimen Volume:** 20 mL

**Specimen Stability Information:** Refrigerated (preferred) <24 hours/Ambient <24 hours

**Additional information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid. An additional 2 to 3 weeks is required to culture amniotic fluid before genetic testing can occur.
3. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

**Specimen Type: Chorionic villi**

**Container/Tube:** 15-mL tube containing 15 mL of transport media

**Specimen Volume:** 20 mg

**Specimen Stability Information:** Refrigerated (preferred) <24 hours/Ambient <24 hours

**Additional Information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.
3. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

**Specimen Type: Prenatal cultured fibroblasts (eg, products of conception), amniocytes, or other confluent cultured cells. This does not include cultured chorionic villi.**

**Container/Tube:** T-25 flask

**Specimen Volume:** 2 Flasks

**Collection Instructions:** Submit confluent cultured cells from another laboratory.

**Specimen Stability Information:** Ambient (preferred) <24 hours/Refrigerated <24 hours)

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
**Additional Information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid or CULFB / Fibroblast Culture for Biochemical or Molecular Testing.
3. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

**Specimen Type: Cultured chorionic villi**

**Container/Tube:** T-25 flasks

**Specimen Volume:** 2 Full flasks



**Collection Instructions:** Submit confluent cultured cells from another laboratory.

**Specimen Stability Information:** Ambient (preferred) <24 hours/Refrigerated <24 hours

**Additional Information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and/or extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing
3. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Current Specimen Stability			
Specimen	Temperature	Time	Special Container
Varies	Ambient (preferred)		
	Frozen		
	Refrigerated		

New Specimen Stability			
Specimen	Temperature	Time	Special Container
Varies	Varies		

**Questions**  
Contact Michelle Rath, Laboratory Resource Coordinator at 800-533-1710.