

## Circulating Immune Complex Panel

**Explanation:** The following new orderable, referred to Arup Laboratories, will be available on April 15, 2025.

**Test ID:** FCICP

**Methods:**

Semi-Quantitative Enzyme-Linked Immunosorbent Assay

**Reference Values:**

Circulating Immune Complex, C1q Binding: 0.0-3.9 ug Eq/mL

Circulating Immune Complex, C3 fragments: Less than or equal to 15 ug Eq/mL

**Specimen Requirements:**

**Collection Container/Tube:**

**Preferred:** Red Top

**Acceptable:** Serum gel

**Submission Container/Tube:** Plastic vial

**Specimen Volume:** 2 mL

**Collection Instructions:**

1. Draw blood in plain, red-top tube(s) or serum gel tube(s).
2. Allow complete clotting of red blood cells (up to 1 hour)
3. Within 30 minutes of clotting completion, centrifuge and aliquot 2 mL into a plastic vial.
3. Freeze immediately.
4. Ship frozen.

**Note:** Critical frozen. Separate specimens must be submitted when multiple tests are ordered.

**Specimen Stability Information:**

Specimen Type	Temperature	Time
Serum	Frozen	14 days

**Fee:**

\$504.20

**CPT Code:**

86332 x 2

**Day(s) Setup:**

Saturday, Monday, and Thursday

**Report Available:**

4 to 15 days

**Questions:**

Contact MCL Referrals Supervisor, Amy Bluhm, at 800-533-1710.