

Mitochondrial Full Genome Analysis, Next-Generation Sequencing (NGS), Varies

Test ID: MITOP

Explanation: On the effective date, the Test Algorithm will be updated to include an additional reflex test when necessary. Additionally, the Genetics Information, Additional Testing Requirements, Specimen Required, and Days Performed sections will be updated as indicated below.

Current Genetics Information

This test includes amplification of the entire mitochondrial genome by long-range polymerase chain reaction followed by sequencing on the next-generation sequencing platform to evaluate for variants within the mitochondrial genome.

New Genetics Information

This test uses next-generation sequencing (NGS) to test for the presence of variants, including 13 protein coding genes, 22 transfer RNA genes, and 2 ribosomal RNA genes, within the mitochondrial genome.

Large deletions within the mitochondrial genome and their locations are determined from the NGS data and confirmed by droplet digital polymerase chain reaction (ddPCR). Large deletion heteroplasmy level is also determined by ddPCR.

This test is only useful for detecting mitochondrial genomic variants. Depletion of mitochondrial DNA levels or variants in mitochondrial genes encoded by the nuclear genome is not within the scope of this test.

Current Testing Algorithm

For skin biopsy or cultured fibroblast specimens, fibroblast culture will be performed at an additional charge. If viable cells are not obtained, the client will be notified.

The following algorithms are available:

- [Epilepsy: Unexplained Refractory and/or Familial Testing Algorithm](#)
- [Neuromuscular Myopathy Testing Algorithm](#)

New Testing Algorithm

Skin biopsy:

If skin biopsy is received, fibroblast culture will be added at an additional charge. If viable cells are not obtained, the client will be notified.

Prenatal specimens:

If an amniotic fluid specimen is received, an amniotic fluid culture will be performed at an additional charge.

If chorionic villi, cultured chorionic villi, or cultured amniocyte specimen is received, a fibroblast culture will be performed at an additional charge.

For any prenatal specimen that is received, maternal cell contamination testing will be performed at an additional charge.

Cord blood:

For cord blood specimens that have an accompanying maternal blood specimen, maternal cell contamination studies will be performed at an additional charge.

The following algorithms are available:

[-Epilepsy: Unexplained Refractory and/or Familial Testing Algorithm](#)

[-Neuromuscular Myopathy Testing Algorithm](#)

Current Reflex Tests			
Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Test	Yes	No

New Reflex Tests			
Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Test	Yes	No
CULAF	Amniotic Fluid Culture/Genetic Test	Yes	No
MATCC	Maternal Cell Contamination, B	Yes	No

Current Additional Testing Requirements
None

New Additional Testing Requirements
For cord blood specimens: Maternal cell contamination (MCC) studies are available. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on both the cord blood and maternal specimens under separate order numbers. Cord blood testing will proceed without MCC studies, but results may be compromised if MCC is present.

Current Specimen Required
<p>Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. For information about testing patients who have received a bone marrow transplant, call 800-533-1710.</p> <p>Submit only 1 of the following specimens:</p> <p>Specimen Type: Whole blood Container/Tube: Preferred: Lavender top (EDTA) or yellow top (ACD) Acceptable: Green top (Sodium heparin) Specimen Volume: 3 mL Collection Instructions: 1. Invert several times to mix blood. 2. Send whole blood specimen in original tube. Do not aliquot. Specimen Stability Information: Ambient 4 days/Refrigerated 4 days/Frozen 4 days Additional Information:</p>

New Specimen Required
<p>Patient Preparation: A previous hematopoietic stem cell transplant from an allogenic donor will interfere with testing. For information about testing patients who have received a hematopoietic stem cell transplant, call 800-533-1710.</p> <p>Submit only 1 of the following specimens:</p> <p>Specimen Type: Whole blood Container/Tube: Preferred: Lavender top (EDTA) or yellow top (ACD) Acceptable: Green top (Sodium heparin) Specimen Volume: 3 mL Collection Instructions: 1. Invert several times to mix blood. 2. Send whole blood specimen in original tube. Do not aliquot. Specimen Stability Information: Ambient 4 days/Refrigerated 4 days/Frozen 4 days Additional Information: 1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.</p>

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.

2. To ensure minimum volume and concentration of DNA is met, the preferred volume of blood must be submitted. Testing may be canceled if DNA requirements are inadequate.

Specimen Type: Cultured fibroblasts

Source: Skin

Container/Tube: T-25 flask

Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a skin biopsy. Cultured cells from a prenatal specimen will not be accepted.

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Skin biopsy

Supplies: Fibroblast Biopsy Transport Media (T115)

Container/Tube: Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.

Specimen Volume: 4-mm Punch

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Snap frozen nerve tissue biopsy

Collection Instructions: Prepare snap frozen tissue biopsy per surgical procedure

Specimen Volume: 0.25 to 0.5 cm

Specimen Stability Information: Frozen

Specimen Type: Muscle tissue biopsy

Supplies: Muscle Biopsy Kit (T541)

2. To ensure minimum volume and concentration of DNA are met, the requested volume must be submitted. Testing may be canceled if DNA requirements are inadequate.

Specimen Type: Cord blood

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Green top (Sodium heparin)

Specimen Volume: 3 mL

Collection Instructions:

1. Invert several times to mix blood.

2. Send cord blood specimen in original tube. Do not aliquot.

Specimen Stability Information: Ambient (preferred) 4 days/Refrigerated 4 days/Frozen 4 days

Additional Information:

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.

2. To ensure minimum volume and concentration of DNA are met, the requested volume must be submitted. Testing may be canceled if DNA requirements are inadequate.

3. While a properly collected cord blood sample may not be at risk for maternal cell contamination, unanticipated complications may occur during collection. Therefore, maternal cell contamination studies are recommended to ensure the test results reflect that of the patient tested and are available at an additional charge. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Saliva

Patient Preparation: Patient should not eat, drink, smoke, or chew gum 30 minutes prior to collection.

Supplies: Saliva Swab Collection Kit (T786)

Specimen Volume: 2 Swabs

Collection Instructions: Collect and send specimen per kit instructions.

Specimen Stability Information: Ambient (preferred) 30 days/Refrigerated 30 days

Additional Information: Saliva specimens are acceptable but not recommended. Due to lower quantity/quality of DNA yielded from saliva, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

Specimen Type: Skin biopsy

Supplies: Fibroblast Biopsy Transport Media (T115)

Container/Tube: Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.

Specimen Volume: 4-mm Punch

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

Specimen Volume: 20 to 80 mg
Collection Instructions: Prepare and transport specimen per instructions in [Muscle Biopsy Specimen Preparation](#).
Specimen Stability Information: Frozen (preferred) <24 hours/Ambient <24 hours/Refrigerated <24 hours
Additional Information: Specimens are preferred to be received within 24 hours of collection. Extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

Specimen Type: Blood spot
Supplies: Card-Blood Spot Collection (Filter Paper) (T493)
Container/Tube:
Preferred: Collection card (Whatman Protein Saver 903 Paper)
Acceptable: PerkinElmer 226 filter paper or blood spot collection card
Specimen Volume: 2 to 5 Blood spots
Collection Instructions:
 1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect a Dried Blood Spot Sample](#).
 2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.
 3. Do not expose specimen to heat or direct sunlight.
 4. Do not stack wet specimens.
 5. Keep specimen dry
Specimen Stability Information: Ambient (preferred)/Refrigerated
Additional Information:
 1. Blood spot specimens are acceptable but not recommended. Multiple extractions will be required to obtain sufficient yield for supplemental analysis, and there is significant risk for test failure due to insufficient DNA.
 2. Due to lower concentration of DNA yielded from blood spot, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.
 3. For collection instructions, see [Blood Spot Collection Instructions](#)
 4. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)
 5. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

Specimen Type: Extracted DNA
Container/Tube:
Preferred: Screw Cap Micro Tube, 2 mL with skirted conical base

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Cultured fibroblasts

Source: Skin

Container/Tube: T-25 flask

Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a tissue biopsy.

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Tissue biopsy

Supplies: Hank's Solution (T132)

Container/Tube: Sterile container with sterile Hank's balanced salt solution, Ringer's solution, or normal saline

Specimen Volume: 0.5 to 3 cm(3) or larger

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur

Specimen Type: Snap frozen nerve tissue biopsy

Collection Instructions: Prepare snap frozen tissue biopsy per surgical procedure

Specimen Volume: 0.25 to 0.5 cm

Specimen Stability Information: Frozen

Specimen Type: Muscle tissue biopsy

Supplies: Muscle Biopsy Kit (T541)

Specimen Volume: 20 to 80 mg

Collection Instructions: Prepare and transport specimen per instructions in [Muscle Biopsy Specimen Preparation](#).

Specimen Stability Information: Frozen (preferred) <24 hours/Ambient <24 hours/Refrigerated <24 hours

Additional Information: Specimens are preferred to be received within 24 hours of collection. Extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

Specimen Type: Blood spot

Supplies: Card-Blood Spot Collection (Filter Paper) (T493)

Acceptable: Matrix tube, 1 mL

Collection Instructions:

1. The preferred volume is at least 100 mcL at a concentration of 75 ng/mcL.

2. Include concentration and volume on tube.

Specimen Stability Information: Frozen (preferred) 1 year/Ambient/Refrigerated

Additional Information: DNA must be extracted in a CLIA-certified laboratory or equivalent and must be extracted from a specimen type listed as acceptable for this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

Container/Tube:

Preferred: Collection card (Whatman Protein Saver 903 Paper)

Acceptable: PerkinElmer 226 filter paper or blood spot collection card

Specimen Volume: 2 to 5 Blood spots

Collection Instructions:

1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect a Dried Blood Spot Sample](#).

2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.

3. Do not expose specimen to heat or direct sunlight.

4. Do not stack wet specimens.

5. Keep specimen dry

Specimen Stability Information: Ambient (preferred)/Refrigerated

Additional Information:

1. Blood spot specimens are acceptable but not recommended. Multiple extractions will be required to obtain sufficient yield for supplemental analysis, and there is significant risk for test failure due to insufficient DNA.

2. Due to lower concentration of DNA yielded from blood spot, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

3. For collection instructions, see [Blood Spot Collection Instructions](#)

4. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)

5. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

Specimen Type: Extracted DNA

Container/Tube:

Preferred: Screw Cap Micro Tube, 2 mL with skirted conical base

Acceptable: Matrix tube, 1 mL

Collection Instructions:

1. The preferred volume is at least 100 mcL at a concentration of 75 ng/mcL.

2. Include concentration and volume on tube.

Specimen Stability Information: Frozen (preferred) 1 year/Ambient/Refrigerated

Additional Information: DNA must be extracted in a CLIA-certified laboratory or equivalent and must be extracted from a specimen type listed as acceptable for this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

PRENATAL SPECIMENS

Due to its complexity, consultation with the laboratory is required for all prenatal testing; call 800-533-1710 to speak to a genetic counselor.

Specimen Type: Amniotic fluid

Container/Tube: Amniotic fluid container

Specimen Volume: 20 mL

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information: Specimen will only be tested after culture.

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid. An additional 2 to 3 weeks are required to culture amniotic fluid before genetic testing can occur.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Confluent cultured amniocytes

Container/Tube: T-25 flask

Specimen Volume: 2 Full flasks

Collection Instructions: Submit confluent cultured amniocytes from another laboratory

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Chorionic villi

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
Container/Tube: 15-mL tube containing 15 mL of transport media

Specimen Volume: 20 mg

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information: Specimen will only be tested after culture.

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.



3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Cultured chorionic villi
Container/Tube: T-25 flasks
Specimen Volume: 2 Full flasks
Collection Instructions: Submit confluent cultured cells from another laboratory
Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours
Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Current Days Performed
Monday

New Days Performed
Varies

Questions
Contact Michelle Rath, Laboratory Resource Coordinator at 800-533-1710.