

## **NEW REFERRAL TEST**

Notification Date: August 4, 2025 Effective Date: August 18, 2025

# IA2 Autoantibodies

**Explanation:** The following new orderable, referred to Esoterix Endocrinology, will be available on August 18, 2025.

Test ID: FIAAB

### Methods:

Enzyme-linked Immunosorbent Assay (ELISA)

#### **Reference Values:**

Negative: <7.5 U/mL

Positive: > or = 7.5 U/mL

### **Specimen Requirements:**

**Collection Container/Tube:** 

Preferred: Serum gel Acceptable: Red top

Submission Container/tube: Plastic vial

Specimen Volume: 1 mL Collection Instructions:

- 1. Centrifuge and aliquot 1 mL of serum into a plastic vial.
- 2. Freeze immediately after separation and send frozen.

Minimum Volume: 0.5 mL

# **Specimen Stability Information:**

Specimen Type	Temperature	Time	
Serum	Frozen	107 days	
	Ambient	72 hours	
	Refrigerated	72 hours	

C	P1	Γ(	C	n	d	6	•

86341

Day(s) Setup: Varies Report Available: 7-14 days

## **Questions**

Contact MCS Referrals Supervisors Amy Bluhm or Heather Hilgart at 800-533-1710.