

Histamine Plasma

Test ID: FHSPL

Explanation: Collection instruction for the processing of specimens for Test ID FHSPL-Histamine Plasma, referred to Eurofins Viracor is effective immediately.

Current Specimen Required	New Specimen Required
<p>Specimen Type: Plasma Collection Container/Tube: Lavender top (EDTA) Submission Container/Tube: Plastic vial Specimen Volume: 3 mL</p> <p>Collection Instructions: Draw 3 mL blood in a lavender-top (EDTA) tube(s). Cool immediately on ice. Centrifuge at 1500 rpm for 10 minutes at 4 degrees C. The centrifugation should be performed within 20 minutes of collection. Carefully remove 1 mL of EDTA plasma from the upper part of the tube. Freeze and send frozen in a plastic vial.</p>	<p>Specimen Type: Plasma Collection Container/Tube: Lavender top (EDTA) Submission Container/Tube: Plastic vial Specimen Volume: Plasma 1 mL</p> <p>Collection Instructions: 1. Draw 3 mL blood in a lavender-top (EDTA) tube(s). 2. Cool immediately on ice. 3. Centrifugation should be performed within 20 minutes of collection at 900 rpm for 10 minutes at 4 degrees C. 4. Carefully remove 1 mL of EDTA plasma from the upper part of the tube. 5. Send frozen.</p>

Questions:

Contact MCS Referrals Supervisors Amy Bluhm or Heather Hilgart at 800-533-1710.