

## SMN1 Gene, Full Gene Analysis, Varies

**Test ID:** SMN1Z

**Explanation:** On the effective date, extracted DNA and cord blood will be acceptable for testing. Formatting of acceptable specimen types will also be standardized. Reflex testing and algorithm will be updated to include new specimen type information.

### Current Algorithm

For skin biopsy or cultured fibroblast specimens, fibroblast culture testing will be performed at an additional charge. If viable cells are not obtained, the client will be notified.

For more information see [Inherited Motor Neuron Disease and Dementia Testing Algorithm](#)

### New Algorithm

**Skin biopsy or cultured fibroblast specimens:**  
For skin biopsy or cultured fibroblast specimens, fibroblast culture testing will be performed at an additional charge. If viable cells are not obtained, the client will be notified.

**Cord blood:**  
For cord blood specimens that have an accompanying maternal blood specimen, maternal cell contamination studies will be performed at an additional charge.

For more information see [Inherited Motor Neuron Disease and Dementia Testing Algorithm](#)

### Current Reflex Tests

Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Testing	Yes	No

### New Reflex Tests

Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Testing	Yes	No
MATCC	Maternal Cell Contamination, B	Yes	No

### Current Specimen Required

**Patient Preparation:** A previous bone marrow transplant from an allogenic donor will interfere with testing. Call 800-533-1710 for instructions for testing patients who have received a bone marrow transplant.

**Submit only 1 of the following specimens:**

**Specimen Type:** Whole blood

**Container/Tube:**

**Preferred:** Lavender top (EDTA) or yellow top (ACD)

**Acceptable:** Any anticoagulant

**Specimen Volume:** 3 mL

**Collection Instructions:**

1. Invert several times to mix blood.
2. Send whole blood specimen in original tube. **Do not aliquot.**

**Specimen Stability Information:** Ambient (preferred)/Refrigerated

**Specimen Type:** Cultured fibroblasts

**Container/Tube:** T-75 or T-25 flask

**Specimen Volume:** 1 Full T-75 or 2 full T-25 flasks

**Specimen Stability Information:** Ambient (preferred)/Refrigerated <24 hours

**Additional Information:** A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

**Supplies:** Fibroblast Biopsy Transport Media (T115)

**Specimen Type:** Skin biopsy

**Container/Tube:** Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.).

**Specimen Volume:** 4-mm punch

**Specimen Stability Information:** Refrigerated (preferred)/Ambient

**Additional Information:** A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

**Supplies:** Card - Blood Spot Collection (Filter Paper) (T493)

**Specimen Type:** Blood spot

**Container/Tube:**

**Preferred:** Collection card (Whatman Protein Saver 903 Paper)

**Acceptable:** PerkinElmer 226 (formerly Ahlstrom 226) filter paper or blood spot collection card

**Specimen Volume:** 5 Blood spots

**Collection Instructions:**

### New Specimen Required

**Patient Preparation:** A previous hematopoietic stem cell transplant from an allogenic donor will interfere with testing. For information about testing patients who have received a hematopoietic stem cell transplant, call 800-533-1710.

**Submit only 1 of the following specimens:**

**Specimen Type:** Whole blood

**Container/Tube:** Lavender top (EDTA) or yellow top (ACD)

**Specimen Volume:** 3 mL

**Collection Instructions:**

1. Invert several times to mix blood.
2. Send whole blood specimen in original tube. **Do not aliquot.**

3. Whole blood collected postnatal from an umbilical cord is also acceptable. See Additional Information

**Specimen Stability Information:** Ambient (preferred) 4 days/Refrigerated 4 days/Frozen 4 days

**Additional Information:**

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.
2. To ensure minimum volume and concentration of DNA are met, the requested volume must be submitted. Testing may be canceled if DNA requirements are inadequate.
3. For postnatal umbilical cord whole blood specimens, maternal cell contamination studies are recommended to ensure test results reflect that of the patient tested. A maternal blood specimen is required to complete maternal cell contamination studies. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on both the cord blood and maternal blood specimens under separate order numbers.

**Specimen Type:** Blood spot

**Supplies:** Card-Blood Spot Collection (Filter Paper) (T493)

**Container/Tube:**

**Preferred:** Collection card (Whatman Protein Saver 903 Paper)

**Acceptable:** PerkinElmer 226 filter paper or blood spot collection card

**Specimen Volume:** 2 to 5 Blood spots

**Collection Instructions:**

1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect a Dried Blood Spot Sample](#).
2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.
3. Do not expose specimen to heat or direct sunlight.
4. Do not stack wet specimens.

1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect Dried Blood Spot Samples](#).

2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.3. Do not expose specimen to heat or direct sunlight.

4. Do not stack wet specimens.

5. Keep specimen dry.

**Specimen Stability Information:** Ambient (preferred)/Refrigerated

**Additional Information:**

1. Due to lower concentration of DNA yielded from blood spot, it is possible that additional specimen may be required to complete testing.

2. For collection instructions, see [Blood Spot Collection Instructions](#)

3. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)

4. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

5. Keep specimen dry.

**Specimen Stability Information:** Ambient (preferred)/Refrigerated

**Additional Information:**

1. Blood spot specimens are acceptable but not recommended. Due to lower quantity/quality of DNA yielded from blood spots, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

2. Due to lower concentration of DNA yielded from blood spot, it is possible that additional specimen may be required to complete testing.

3. For collection instructions, see [Blood Spot Collection Instructions](#)

4. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)

5. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

**Specimen Type:** Skin biopsy

**Supplies:** Fibroblast Biopsy Transport Media (T115)

**Container/Tube:** Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.

**Specimen Volume:** 4-mm Punch

**Specimen Stability Information:** Ambient (preferred) <24 hours/Refrigerated <24 hours

**Additional Information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

**Specimen Type:** Cultured fibroblasts

**Source:** Skin or tissue

**Container/Tube:** T-25 Flask

**Specimen Volume:** 2 Flasks

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
**Collection Instructions:** Submit confluent cultured fibroblast cells from a biopsy. Cultured cells from a prenatal specimen will not be accepted.

**Specimen Stability Information:** Ambient (preferred) <24 hours/Refrigerated <24 hours

**Additional Information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are



required to culture fibroblasts before genetic testing can occur.

**Specimen Type:** Extracted DNA  
**Container/Tube:**  
**Preferred:** Screw Cap Micro Tube, 2mL with skirted conical base  
**Acceptable:** Matrix tube, 1 mL  
**Collection Instructions:**  
1. The preferred volume is at least 100 mcL at a concentration of 75 ng/mcL.  
2. Include concentration and volume on tube.  
**Specimen Stability Information:** Frozen (preferred) 1 year/Ambient/Refrigerated  
**Additional Information:** DNA must be extracted in a CLIA-certified laboratory or equivalent and must be extracted from a specimen type listed as acceptable for this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

Current Specimen Retention
Whole Blood: 2 weeks (if available); Extracted DNA: 3 months

New Specimen Retention
Whole blood: 28 days (if available); Extracted DNA: 3 months, Blood spots: 1 year (if available)

**Questions**  
Contact Melissa Tricker-Klar, Laboratory Resource Coordinator at 800-533-1710.