

Sarcoma Targeted Gene Fusion/Rearrangement Panel, Next-Generation Sequencing, Tumor

Test ID: SARCP

Explanation:

On the effective date, Test ID: SARCP will be obsolete and replaced with the recommended alternative test below.

Recommended Alternative Test:

MayoComplete Targeted RNA Sequencing Panel, Next-Generation Sequencing, Tumor

Test ID: MCRSP

Methodology:

Sequence Capture and Targeted Next-Generation Sequencing (NGS)

Reference Values:

An interpretive report will be provided.

Specimen Requirements:

This assay requires at least 10% tumor nuclei.

-Preferred amount of tumor area with sufficient percent tumor nuclei: tissue 144 mm(2)

-Minimum amount of tumor area: tissue 36 mm(2)

-These amounts are cumulative over up to 10 unstained slides and must have adequate percent tumor nuclei.

-Tissue fixation: 10% neutral buffered formalin, not decalcified

-For specimen preparation guidance, see [Tissue Requirements for Solid Tumor Next-Generation Sequencing](#).

In this document, the sizes are given as 4 mm x 4 mm x 10 slides as preferred: approximate/equivalent to 144 mm(2) and the minimum as 3 mm x 1 mm x 10 slides: approximate/equivalent to 36 mm(2).

Preferred: Submit 3, if available, or 2 of the following specimens.

Acceptable: Submit **at least one** of the following specimens.

Specimen Type: Tissue block

Collection Instructions: Submit a formalin-fixed, paraffin-embedded tissue block with acceptable amount of tumor tissue.

Specimen Type: Tissue slides

Slides: 1 Hematoxylin and eosin-stained and 10 unstained

Collection Instructions:

Submit the following slides:

1 Slide stained with hematoxylin and eosin

AND

10 Unstained, nonbaked slides with 5-micron thick sections of the tumor tissue.

Note: The total amount of required tumor nuclei can be obtained by scraping up to 10 slides from the same block.

Additional Information: Unused unstained slides will not be returned.

Specimen Type: Cytology slides (direct smears or ThinPrep)

Slides: 1 to 3 Slides

Collection Instructions: Submit 1 to 3 slides stained and coverslipped with a total of 5000 nucleated cells (preferred) or at least 3000 nucleated cells (minimum).

Note: Glass coverslips are preferred; plastic coverslips are acceptable but will result in longer turnaround times.

Additional Information: Cytology slides will not be returned. An image of the slides will be stored per regulatory requirements.

Specimen Stability Information:

Specimen Type	Temperature	Time
Varies	Ambient (preferred)	
	Refrigerated	

CPT Code:

81456

Day(s) Performed: Monday through Friday

Report Available: 12-20 days

Questions

Contact Melissa Tricker-Klar, Laboratory Resource Coordinator at 800-533-1710.