

Bartter Syndrome Gene Panel, Varies

Test ID: RBART

Explanation: On the effective date extracted DNA and prenatals: amniotic fluid and chorionic villi will be acceptable for testing. Formatting of acceptable specimen types will also be standardized. Reflex testing and algorithm will be updated to include new specimen type information.

Current Testing Algorithm

None

New Testing Algorithm

Prenatal specimens:

If an amniotic fluid specimen or cultured amniocytes are received, an amniotic fluid culture will be performed at an additional charge.

If a chorionic villi specimen or cultured chorionic villi are received, a fibroblast culture will be performed at an additional charge.

For any prenatal specimen that is received, maternal cell contamination testing will be performed at an additional charge.

Skin biopsy or cultured fibroblast specimens:

For skin biopsy or cultured fibroblast specimens, a fibroblast culture will be performed at an additional charge. If viable cells are not obtained, the client will be notified.

Cord blood:

For cord blood specimens that have an accompanying maternal blood specimen, maternal cell contamination studies will be performed at an additional charge.

Current Ordering Guidance

The genes associated with Gitelman syndrome (SLC12A3) and autosomal dominant familial hypocalciuric hypercalcemia (FHH) (CASR) are not included on this panel. If testing for these disorders and Bartter syndrome on a single panel is desired, order RSCGP / Nephrocalcinosis, Nephrolithiasis, and Renal Electrolyte Imbalance Gene Panel, Varies. It is inappropriate to order both this test and RSCGP on the same patient because the genes on this panel are included on the RSCGP panel.

New Ordering Guidance

The genes associated with Gitelman syndrome (SLC12A3) and autosomal dominant familial hypocalciuric hypercalcemia (FHH, CASR) are not included on this panel. If testing for these disorders and Bartter syndrome on a single panel is desired, order RSCGP / Nephrocalcinosis, Nephrolithiasis, and Renal Electrolyte Imbalance Gene Panel, Varies. It is inappropriate to order both this test and RSCGP on the same patient because the genes on this panel are included on the RSCGP panel.

Targeted testing for familial variants (also called site-specific or known mutations testing) is available for the genes on this panel. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

Customization of this panel and single gene analysis for any gene present on this panel are available. For more information, see CGPH / Custom Gene Panel, Hereditary, Next-Generation Sequencing, Varies.

Targeted testing for familial variants (also called site-specific or known mutations testing) is available for the genes on this panel. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

Customization of this panel and single gene analysis for any gene present on this panel are available. For more information, see CGPH / Custom Gene Panel, Hereditary, Next-Generation Sequencing, Varies. To modify this panel via CGPH, use the Hereditary Renal Conditions disease state for step 1 on the [Custom Gene Ordering Tool](#).

Current Reflex Tests

Test ID	Reporting Name	Available Separately	Always Performed
None			

New Reflex Tests

Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Test	Yes	No
CULAF	Amniotic Fluid Culture/Genetic Test	Yes	No
MATCC	Maternal Cell Contamination, B	Yes	No

Current Specimen Required

Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. Call 800-533-1710 for instructions for testing patients who have received a bone marrow transplant.

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Any anticoagulant

Specimen Volume: 3 mL

Collection Instructions:

1. Invert several times to mix blood.
2. Send whole blood specimen in original tube. **Do not aliquot.**

Specimen Stability Information: Ambient (preferred)/Refrigerated

New Specimen Required

Patient Preparation A previous hematopoietic stem cell transplant from an allogenic donor will interfere with testing. For information about testing patients who have received a hematopoietic stem cell transplant, call 800-533-1710.

Submit only 1 of the following specimens:

Specimen Type: Whole blood

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Specimen Volume: 3 mL

Collection Instructions:

1. Invert several times to mix blood.
 2. Send whole blood specimen in original tube. **Do not aliquot.**
 3. Whole blood collected postnatal from an umbilical cord is also acceptable. See Additional Information
- Specimen Stability Information:** Ambient (preferred) 4 days/Refrigerated 4 days/Frozen 4 days

Additional Information:

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.
2. To ensure minimum volume and concentration of DNA are met, the requested volume must be submitted.

Testing may be canceled if DNA requirements are inadequate.

3. For postnatal umbilical cord whole blood specimens, maternal cell contamination studies are recommended to ensure test results reflect that of the patient tested. A maternal blood specimen is required to complete maternal cell contamination studies. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on both the cord blood and maternal blood specimens under separate order numbers.

Specimen Type: Cultured fibroblasts

Source: Skin or tissue

Container/Tube: T-25 flask

Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a skin or tissue biopsy. Cultured cells from a prenatal specimen will not be accepted.

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Extracted DNA

Container/Tube:

Preferred: Screw Cap Micro Tube, 2mL with skirted conical base

Acceptable: Matrix tube, 1 mL

Collection Instructions:

1. The preferred volume is at least 100 mcL at a concentration of 75 ng/mcL.
2. Include concentration and volume on tube.

Specimen Stability Information: Frozen (preferred) 1 year/Ambient/Refrigerated

Additional Information: DNA must be extracted in a CLIA-certified laboratory or equivalent and must be extracted from a specimen type listed as acceptable for this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

Prenatal Specimens

Due to its complexity, consultation with the laboratory is required for all prenatal testing; call 800-533-1710 to speak to a genetic counselor.

Specimen Type: Amniotic fluid

Container/Tube: Amniotic fluid container

Specimen Volume: 20 mL

Specimen Stability Information: Ambient (preferred)
<24 hours/Refrigerated <24 hours

Additional Information: Specimen will only be tested after culture.

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid. An additional 2 to 3 weeks are required to culture amniotic fluid before genetic testing can occur.

3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Confluent cultured amniocytes
This does not include cultured chorionic villi.

Container/Tube: T-25 flask

Specimen Volume: 2 Full flasks

Collection Instructions: Submit confluent cultured cells from another laboratory.

Specimen Stability Information: Ambient (preferred)
<24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing.

3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Chorionic villi

Container/Tube: 15-mL tube containing 15 mL of transport media

Specimen Volume: 20 mg

Specimen Stability Information: Ambient (preferred)
<24 hours/Refrigerated <24 hours

Additional Information: Specimen will only be tested after culture.

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

--

Specimen Type: Cultured chorionic villi
Container/Tube: T-25 flasks
Specimen Volume: 2 Full flasks
Collection Instructions: Submit confluent cultured cells from another laboratory
Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours
Additional Information:
1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Current Specimen Retention Time
Whole blood: 2 weeks (if available); Extracted DNA: 3 months

New Specimen Retention Time
Whole blood: Refrigerate; Extracted DNA: Frozen

Questions

Contact Melissa Tricker-Klar, Laboratory Resource Coordinator at 800-533-1710.