

Panel to Whole Exome Sequencing Reflex Test, Varies

Test ID: WESPR

Explanation: On the effective date, extracted DNA, and prenatal: amniotic fluid and chorionic villi will be acceptable for testing. Formatting of acceptable specimen types will also be standardized. Reflex testing and algorithm will be updated to include new specimen type information.

Current Testing Algorithm

If a cord blood specimen is received, maternal cell contamination testing will be added and performed at an additional charge.

For skin biopsy or cultured fibroblast specimens, fibroblast culture will be performed at an additional charge. If viable cells are not obtained, the client will be notified.

For more information see [Epilepsy: Unexplained Refractory and/or Familial Testing Algorithm](#)

New Testing Algorithm

Prenatal specimens:

If an amniotic fluid specimen or cultured amniocytes are received, an amniotic fluid culture will be performed at an additional charge.

If a chorionic villi specimen or cultured chorionic villi are received, a fibroblast culture will be performed at an additional charge.

For any prenatal specimen that is received, maternal cell contamination testing will be performed at an additional charge.

Skin biopsy or cultured fibroblast specimens:

For skin biopsy or cultured fibroblast specimens, a fibroblast culture will be performed at an additional charge. If viable cells are not obtained, the client will be notified.

Cord blood:

For cord blood specimens that have an accompanying maternal blood specimen, maternal cell contamination studies will be performed at an additional charge.

Current Ordering Guidance

The American College of Medical Genetics and Genomics recommends that whole exome sequencing be considered as a first-tier or second-tier test for patients with one or more congenital anomalies, or developmental delay, or intellectual disability with onset prior to age 18 years.(1)

This test is only appropriate for patients who have had hereditary panel testing performed on a postnatal sample via next-generation sequencing (NGS) utilizing the Integrated DNA Technologies chemistry performed by Mayo Clinic Laboratories. To confirm

New Ordering Guidance

The American College of Medical Genetics and Genomics recommends that whole exome sequencing be considered as a first-tier or second-tier test for patients with one or more congenital anomalies, or developmental delay, or intellectual disability with onset prior to age 18 years.(1)

This test is only appropriate for patients who have had hereditary gene panel testing performed on a postnatal sample via next-generation sequencing (NGS) utilizing the Integrated DNA Technologies chemistry performed by Mayo Clinic Laboratories. To confirm that this test is

that this test is possible for a specific patient, contact the laboratory at 800-533-1710.

If the patient has not had an appropriate test previously performed by Mayo Clinic Laboratories that can be reflexed but whole exome sequencing is desired, order either WESDX / Whole Exome Sequencing for Hereditary Disorders, Varies or WESMT / Whole Exome and Mitochondrial Genome Sequencing, Varies. If whole genome sequencing is desired, order WGSDX / Whole Genome Sequencing for Hereditary Disorders, Varies. A new specimen may be required.

This test is for affected patients (probands) only. It is possible to add family member comparators. For family member specimens being sent as comparators, order CMPRE / Family Member Comparator Specimen for Exome Sequencing, Varies. If WESPR is ordered on a family member comparator, this test will be canceled and CMPRE added as the appropriate test.

This test cannot support detection of deep intronic variants, trinucleotide repeat variants, or variants in the mitochondrial genome.

If separate mitochondrial genome testing is needed, order MITOP / Mitochondrial Full Genome Analysis, Next-Generation Sequencing (NGS), Varies

This test is **not appropriate** for identification of somatic variants in solid tumors. If this testing is needed, order MCSTP / MayoComplete Solid Tumor Panel, Next-Generation Sequencing, Tumor. A new specimen may be required.

This testing does not provide genotyping of patients for pharmacogenomic purposes. For an assessment for genes with strong drug-gene associations, order PGXQP / Focused Pharmacogenomics Panel, Varies. A new specimen may be required.

Targeted testing for familial variants (also called site-specific or known variant testing) is available for variants identified by this test. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

Prenatal specimens (amniocentesis or chorionic villi) are not currently accepted for this test.

possible for a specific patient, contact the laboratory at 800-533-1710.

If the patient has not had an appropriate test previously performed by Mayo Clinic Laboratories but whole exome sequencing is desired, order either WESDX / Whole Exome Sequencing for Hereditary Disorders, Varies or WESMT / Whole Exome and Mitochondrial Genome Sequencing, Varies. If whole genome sequencing is desired, order WGSDX / Whole Genome Sequencing for Hereditary Disorders, Varies. A new specimen may be required.

This test is only for affected patients (probands). It is possible to add family member comparators. For family member specimens being sent as comparators, order CMPRE / Family Member Comparator Specimen for Exome Sequencing, Varies. If WESPR is ordered on a family member comparator, this test will be canceled and CMPRE added as the appropriate test.

This test cannot support detection of deep intronic variants, trinucleotide repeat variants, or variants in the mitochondrial genome.

-If separate mitochondrial genome testing is needed, order MITOP / Mitochondrial Full Genome Analysis, Next-Generation Sequencing (NGS), Varies.

-For detection of single nucleotide variants, small insertions and deletions, copy number variants, deep intronic variants, trinucleotide repeat variants, and mitochondrial genome variants, order WGSDX / Whole Genome Sequencing for Hereditary Disorders, Varies.

This test is **not appropriate** for identification of somatic mutations in solid tumors. If this testing is needed, order MCSTP / MayoComplete Solid Tumor Panel, Next-Generation Sequencing, Tumor. A new specimen may be required.

This test does not provide genotyping of patients for pharmacogenomic purposes. For an assessment for genes with strong drug-gene associations, order PGXQP / Focused Pharmacogenomics Panel, Varies. A new specimen may be required.

Targeted testing for familial variants (also called site-specific or known variant testing) is available for variants identified by this test. See FMTT / Familial Variant, Targeted Testing, Varies. To obtain more information about this testing option, call 800-533-1710.

Current Reflex Tests			
Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Test	Yes	No
MATCC	Maternal Cell Contamination, B	Yes	No
G237	Number of Comparators for WESPR	No	No

New Reflex Tests			
Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Test	Yes	No
MATCC	Maternal Cell Contamination, B	Yes	No
G237	Number of Comparators for WESPR	No	No
CULAF	Amniotic Fluid Culture/Genetic Test	Yes	No

Current Specimen Required
<p>For most patients, a new specimen submission will not be required. Testing can be performed using stored DNA from the original whole exome sequencing test. To order testing on the stored specimen, see Additional Testing Requirements.</p> <p>Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. Call 800-533-1710 for instructions for testing patients who have received a bone marrow transplant.</p> <p>Submit only 1 of the following specimens:</p> <p>Specimen Type: Whole blood Container/Tube: Lavender top (EDTA) or yellow top (ACD) Specimen Volume: 3 mL Collection Instructions:</p> <ol style="list-style-type: none"> 1. Invert several times to mix blood. 2. Send whole blood specimen in original tube. Do not aliquot. 3. Whole blood collected postnatal from an umbilical cord is also acceptable. See Additional Information <p>Specimen Stability Information: Ambient 4 days/Refrigerated 4 days/Frozen 4 days Additional Information:</p> <ol style="list-style-type: none"> 1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for samples received after 4 days and DNA yield will be evaluated to determine if testing may proceed. 2. To ensure minimum volume and concentration of DNA is met, the preferred volume of blood must be submitted. Testing may be canceled if DNA requirements are inadequate. 3. For postnatal umbilical cord whole blood specimens, maternal cell contamination studies are recommended to ensure test results reflect that of the patient tested. A maternal blood specimen is required to complete maternal cell contamination studies. Order MATCC / Maternal Cell Contamination, Molecular Analysis,

New Specimen Required
<p>For most patients, a new specimen submission will not be required. Testing can be performed using stored DNA from the original whole exome sequencing test. To order testing on the stored specimen, see Additional Testing Requirements.</p> <p>Patient Preparation: A previous hematopoietic stem cell transplant from an allogenic donor will interfere with testing. For information about testing patients who have received a hematopoietic stem cell transplant, call 800-533-1710.</p> <p>Submit only 1 of the following specimens:</p> <p>Specimen Type: Whole blood Container/Tube: Preferred: Lavender top (EDTA) or yellow top (ACD) Acceptable: Green top (sodium heparin) Specimen Volume: 3 mL Collection Instructions:</p> <ol style="list-style-type: none"> 1. Invert several times to mix blood. 2. Send whole blood specimen in original tube. Do not aliquot. 3. Whole blood collected postnatal from an umbilical cord is also acceptable. See Additional Information <p>Specimen Stability Information: Ambient (preferred) 4 days/Refrigerated 4 days/Frozen 4 days Additional Information:</p> <ol style="list-style-type: none"> 1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed. 2. To ensure minimum volume and concentration of DNA are met, the requested volume must be submitted. Testing may be canceled if DNA requirements are inadequate. 3. For postnatal umbilical cord whole blood specimens, maternal cell contamination studies are recommended to ensure test results reflect that of the patient tested. A maternal blood specimen is required to complete

Varies on both the cord blood and maternal blood specimens under separate order numbers.

Specimen Type: Skin biopsy

Supplies: Fibroblast Biopsy Transport Media (T115)

Container/Tube: Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.

Specimen Volume: 4-mm punch

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and/or extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

Specimen Type: Cultured fibroblasts

Source: Skin

Container/Tube: T-25 flask

Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a skin biopsy from another laboratory. **Cultured cells from a prenatal specimen will not be accepted.**

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and/or extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks is required to culture fibroblasts before genetic testing can occur.

Specimen Type: Blood spot

Supplies: Card-Blood Spot Collection (Filter Paper) (T493)

Container/Tube:

Preferred: Collection card (Whatman Protein Saver 903 Paper)

Acceptable: PerkinElmer 226 (formally Ahlstrom 226) filter paper or blood spot collection card

Specimen Volume: 2 to 5 Blood spots

Collection Instructions:

1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect Dried Blood Spot Samples](#).

maternal cell contamination studies. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on both the cord blood and maternal blood specimens under separate order numbers.

Specimen Type: Saliva

Patient Preparation: Patient should not eat, drink, smoke, or chew gum 30 minutes prior to collection.

Supplies:

DNA Saliva Kit High Yield (T1007)

Saliva Swab Collection Kit (T786)

Container/Tube:

Preferred: High-yield DNA saliva kit

Acceptable: Saliva swab

Specimen Volume: 1 Tube if using T1007 or 2 swabs if using T786

Collection Instructions: Collect and send specimen per kit instructions.

Specimen Stability Information: Ambient (preferred) 30 days/Refrigerated 30 days

Additional Information: Saliva specimens are acceptable but not recommended. Due to lower quantity/quality of DNA yielded from saliva, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

Specimen Type: Blood spot

Supplies: Card-Blood Spot Collection (Filter Paper) (T493)

Container/Tube:

Preferred: Collection card (Whatman Protein Saver 903 Paper)

Acceptable: PerkinElmer 226 filter paper or blood spot collection card

Specimen Volume: 2 to 5 Blood spots

Collection Instructions:

1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect a Dried Blood Spot Sample](#).
2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.
3. Do not expose specimen to heat or direct sunlight.
4. Do not stack wet specimens.
5. Keep specimen dry.

Specimen Stability Information: Ambient (preferred)/Refrigerated

Additional Information:

1. Blood spot specimens are acceptable but not recommended. Multiple extractions will be required to obtain sufficient yield for supplemental analysis, and there is significant risk for test failure due to insufficient DNA.
2. Due to lower concentration of DNA yielded from blood spot, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be

2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.

3. Do not expose specimen to heat or direct sunlight.

4. Do not stack wet specimens.

5. Keep specimen dry.

Specimen Stability Information: Ambient (preferred)/Refrigerated

Additional Information:

1. Blood spot specimens are acceptable, but not recommended. Multiple extractions will be required to obtain sufficient yield for supplemental analysis, and there is significant risk for test failure due to insufficient DNA.

2. Due to lower concentration of DNA yielded from blood spot, it is possible that additional specimen may be required to complete testing.

3. For collection instructions, see [Blood Spot Collection Instructions](#)

4. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)

5. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

Specimen Type: Saliva

Patient Preparation: Patient should not eat, drink, smoke, or chew gum 30 minutes prior to collection.

Supplies:

DNA Saliva Kit High Yield (T1007)

Saliva Swab Collection Kit (T786)

Container/Tube:

Preferred: High-yield DNA saliva kit

Acceptable: Saliva swab

Specimen Volume: 1 Tube if using T1007 or 2 swabs if using T786

Collection Instructions: Collect and send specimen per kit instructions.

Specimen Stability Information: Ambient (preferred) 30 days/Refrigerated 30 days

Additional Information: Saliva specimens are acceptable but not recommended. Due to lower quantity/quality of DNA yielded from saliva, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

3. For collection instructions, see [Blood Spot Collection Instructions](#)

4. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)

5. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

Specimen Type: Skin biopsy

Supplies: Fibroblast Biopsy Transport Media (T115)

Container/Tube: Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.

Specimen Volume: 4-mm Punch

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Cultured fibroblasts

Source: Skin or tissue

Container/Tube: T-25 flask

Specimen Volume: 2 Flasks

Collection Instructions: Submit confluent cultured fibroblast cells from a skin or tissue biopsy.

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.

2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

Specimen Type: Extracted DNA

Container/Tube:

Preferred: Screw Cap Micro Tube, 2 mL with skirted conical base

Acceptable: Matrix tube, 1 mL

Collection Instructions:

1. The preferred volume is at least 100 mcL at a concentration of 75 ng/mcL.

2. Include concentration and volume on tube.

Specimen Stability Information: Frozen (preferred) 1 year/Ambient/Refrigerated

Additional Information: DNA must be extracted in a CLIA-certified laboratory or equivalent and must be extracted from a specimen type listed as acceptable for

this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

Prenatal Specimens

Due to its complexity, consultation with the laboratory is required for all prenatal testing; call 800-533-1710 to speak to a genetic counselor.

Specimen Type: Amniotic fluid

Container/Tube: Amniotic fluid container

Specimen Volume: 20 mL

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information: Specimen will only be tested after culture.

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULAF / Culture for Genetic Testing, Amniotic Fluid. An additional 2 to 3 weeks are required to culture amniotic fluid before genetic testing can occur.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Confluent cultured amniocytes

Container/Tube: T-25 flask

Specimen Volume: 2 Full flasks

Collection Instructions: Submit confluent cultured amniocytes from another laboratory.

Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours

Additional Information:

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing.
3. **All prenatal specimens must be accompanied by a maternal blood specimen;** order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Specimen Type: Chorionic villi

Container/Tube: 15-mL tube containing 15 mL of transport media

Specimen Volume: 20 mg

--

<p>Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours</p> <p>Additional Information: Specimen will only be tested after culture.</p> <ol style="list-style-type: none"> 1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed. 2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur. 3. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen. <p>Specimen Type: Cultured chorionic villi</p> <p>Container/Tube: T-25 flasks</p> <p>Specimen Volume: 2 Full flasks</p> <p>Collection Instructions: Submit confluent cultured cells from another laboratory</p> <p>Specimen Stability Information: Ambient (preferred) <24 hours/Refrigerated <24 hours</p> <p>Additional Information:</p> <ol style="list-style-type: none"> 1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed. 2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. 3. All prenatal specimens must be accompanied by a maternal blood specimen; order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on the maternal specimen.

Current Report Available
10 weeks

New Report Available
56 to 70 days

Current Specimen Retention Time
Whole blood: 2 weeks (if available); Extracted DNA: 3 months; Blood spots, saliva, cultured fibroblasts, skin biopsy, cord blood: 1 month

New Specimen Retention Time
Whole blood: 28 days (if available); Saliva: 30 days (if available); Extracted DNA: 3 months; Blood spots: 1 year (if available)

Questions

Contact Melissa Tricker-Klar, Laboratory Resource Coordinator at 800-533-1710.