

## Hereditary Angioedema Focused Gene Panel, Next-Generation Sequencing, Varies

**Test ID:** GNANG

### Explanation:

On the effective date, GNANG will be obsolete. The recommended alternative is below.

### Recommended Alternative Test:

## Angioedema and Complement Disorders Gene Panel, Varies

**Test ID:** AECDP

### Methodology:

Sequence Capture and Amplicon-Based Next-Generation Sequencing (NGS) followed by Droplet Digital Polymerase Chain Reaction (ddPCR)/Quantitative Real-Time Polymerase Chain Reaction (qPCR) and Sanger Sequencing as needed

### Genetics Information:

This test utilizes next-generation sequencing to detect single nucleotide and copy number variants in 28 genes associated with a complement component deficiency or hereditary angioedema: *ANGPT1*, *C1QA*, *C1QB*, *C1QC*, *C1R*, *C1S*, *C2*, *C3*, *C5*, *C6*, *C7*, *C8A*, *C8B*, *C9*, *CD46*, *CD55*, *CD59*, *CFB*, *CFD*, *CFH*, *CFI*, *CFP*, *F12*, *KNG1*, *MASP2*, *PLG*, *SERPING1*, and *THBD*. See [Targeted Genes and Methodology Details for Angioedema and Complement Disorders Gene Panel](#) and Method Description for additional details.

Identification of a disease-causing variant may assist with diagnosis, prognosis, clinical management, recurrence risk assessment, familial screening, and genetic counseling for complement component deficiencies or hereditary angioedema.

### Reflex Tests:

Test ID	Reporting Name	Available Separately	Always Performed
CULFB	Fibroblast Culture for Genetic Test	Yes	No
MATCC	Maternal Cell Contamination, B	Yes	No

### Reference Values:

An interpretive report will be provided.

## **Specimen Requirements:**

**Submit only 1 of the following specimens:**

**Specimen Type:** Whole blood

**Container/Tube:** Lavender top (EDTA) or yellow top (ACD)

**Specimen Volume:** 3 mL

**Collection Instructions:**

1. Invert several times to mix blood.
2. Send whole blood specimen in original tube. **Do not aliquot.**
3. Whole blood collected postnatal from an umbilical cord is also acceptable. See Additional Information.

**Specimen Stability Information:** Ambient (preferred) 4 days/Refrigerated 4 days/Frozen 4 days

**Additional Information:**

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.
2. To ensure minimum volume and concentration of DNA are met, the requested volume must be submitted. Testing may be canceled if DNA requirements are inadequate.
3. For postnatal umbilical cord whole blood specimens, maternal cell contamination studies are recommended to ensure test results reflect that of the patient tested. A maternal blood specimen is required to complete maternal cell contamination studies. Order MATCC / Maternal Cell Contamination, Molecular Analysis, Varies on both the cord blood and maternal blood specimens under separate order numbers.

**Specimen Type:** Skin biopsy

**Supplies:** Fibroblast Biopsy Transport Media (T115)

**Container/Tube:** Sterile container with any standard cell culture media (eg, minimal essential media, RPMI 1640). The solution should be supplemented with 1% penicillin and streptomycin.

**Specimen Volume:** 4-mm Punch

**Specimen Stability Information:** Ambient (preferred) <24 hours/Refrigerated <24 hours

**Additional Information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical or Molecular Testing. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

**Specimen Type:** Cultured fibroblasts

**Source:** Skin

**Container/Tube:** T-25 flask

**Specimen Volume:** 2 Flasks

**Collection Instructions:** Submit confluent cultured fibroblast cells from a skin biopsy. Cultured cells from a prenatal specimen will not be accepted.

**Specimen Stability Information:** Ambient (preferred) <24 hours/Refrigerated <24 hours

**Additional Information:**

1. Specimens are preferred to be received within 24 hours of collection. Culture and extraction will be attempted for specimens received after 24 hours and will be evaluated to determine if testing may proceed.
2. A separate culture charge will be assessed under CULFB / Fibroblast Culture for Biochemical and Molecular Testing, Tissue. An additional 3 to 4 weeks are required to culture fibroblasts before genetic testing can occur.

**Specimen Type:** Extracted DNA

**Container/Tube:**

**Preferred:** Screw Cap Micro Tube, 2 mL with skirted conical base

**Acceptable:** Matrix tube, 1 mL

**Collection Instructions:**

1. The preferred volume is at least 100 mcL at a concentration of 75 ng/mcL.
2. Include concentration and volume on tube.

**Specimen Stability Information:** Frozen (preferred) 1 year/Ambient/Refrigerated

**Additional Information:** DNA must be extracted in a CLIA-certified laboratory, or equivalent, and must be extracted from a specimen type listed as acceptable for this test (including applicable anticoagulants). Our laboratory has experience with Chemagic, Puregene, Autopure, MagnaPure, and EZ1 extraction platforms and cannot guarantee that all extraction methods are compatible with this test. If testing fails, one repeat will be attempted, and if unsuccessful, the test will be reported as failed and a charge will be applied. If applicable, specific gene regions that were unable to be interrogated due to DNA quality will be noted in the report.

**Specimen Type:** Bone marrow aspirate

**Container/Tube:**

**Preferred:** Lavender top (EDTA)

**Acceptable:** Yellow top (ACD)

**Specimen Volume:** 2 mL

**Collection Instructions:**

1. Invert several times to mix bone marrow.
2. Label specimen as bone marrow.
3. Send bone marrow specimen in original tube. **Do not aliquot.**

**Specimen Stability Information:** Ambient 4 days/Refrigerated 4 days/Frozen 4 days

**Additional Information:**

1. Specimens are preferred to be received within 4 days of collection. Extraction will be attempted for specimens received after 4 days, and DNA yield will be evaluated to determine if testing may proceed.
2. To ensure minimum volume and concentration of DNA is met, the required volume must be submitted. Testing may be canceled if DNA requirements are inadequate.

**Specimen Type:** Blood spot

**Supplies:** Card-Blood Spot Collection (Filter Paper) (T493)

**Container/Tube:**

**Preferred:** Collection card (Whatman Protein Saver 903 Paper)

**Acceptable:** PerkinElmer 226 filter paper or blood spot collection card

**Specimen Volume:** 2 to 5 Blood spots

**Collection Instructions:**

1. An alternative blood collection option for a patient older than 1 year is a fingerstick. For detailed instructions, see [How to Collect a Dried Blood Spot Sample](#).
2. Let blood dry on the filter paper at ambient temperature in a horizontal position for a minimum of 3 hours.
3. Do not expose specimen to heat or direct sunlight.
4. Do not stack wet specimens.
5. Keep specimen dry.

**Specimen Stability Information:** Ambient (preferred)/Refrigerated

**Additional Information:**

1. Blood spot specimens are acceptable but not recommended. Multiple extractions will be required to obtain sufficient yield for supplemental analysis, and there is significant risk for test failure due to insufficient DNA.
2. Due to lower concentration of DNA yielded from blood spot, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to

be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

3. For collection instructions, see [Blood Spot Collection Instructions](#)

4. For collection instructions in Spanish, see [Blood Spot Collection Card-Spanish Instructions](#) (T777)

5. For collection instructions in Chinese, see [Blood Spot Collection Card-Chinese Instructions](#) (T800)

**Specimen Type:** Saliva

**Patient Preparation:** Patient should not eat, drink, smoke, or chew gum 30 minutes prior to collection.

**Supplies:**

DNA Saliva Kit High Yield (T1007)

Saliva Swab Collection Kit (T786)

**Container/Tube:**

**Preferred:** High-yield DNA saliva kit

**Acceptable:** Saliva swab

**Specimen Volume:** 1 Tube if using T1007 or 2 swabs if using T786

**Collection Instructions:** Collect and send specimen per kit instructions.

**Specimen Stability Information:** Ambient (preferred) 30 days/Refrigerated 30 days

**Additional Information:** Saliva specimens are acceptable but not recommended. Due to lower quantity/quality of DNA yielded from saliva, some aspects of the test may not perform as well as DNA extracted from a whole blood sample. When applicable, specific gene regions that were unable to be interrogated will be noted in the report. Alternatively, additional specimen may be required to complete testing.

#### **Specimen Stability Information:**

Specimen Type	Temperature	Time
Varies	Varies	

**CPT Code:**

81479

**Day(s) Performed:** Varies

**Report Available:** 28 to 42 days

#### **Questions**

Contact Melissa Lonzo, Laboratory Resource Coordinator, at 800-533-1710.